|     |                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
|-----|--------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| en  | <b>Instructions for use/Technical description</b><br>Forceps                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
| USA | Note for U.S. users<br>This Instructions for Use is NOT intended for United States users. Please discard. The Instructions for Use for United States users can be obtained by visiting our website at <a href="http://www.aesculapusa.com">www.aesculapusa.com</a> . If you wish to obtain a paper copy of the Instructions for Use, you may request one by contacting your local Aesculap representative or Aesculap's customer service at 1-800-282-9000. A paper copy will be provided to you upon request at no additional cost. |
| de  | <b>Gebrauchsanweisung/Technische Beschreibung</b><br>Pinzetten                                                                                                                                                                                                                                                                                                                                                                                                                                                                       |
| fr  | <b>Mode d'emploi/Description technique</b><br>Pincettes                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
| es  | <b>Instrucciones de manejo/Descripción técnica</b><br>Pinzas                                                                                                                                                                                                                                                                                                                                                                                                                                                                         |
| it  | <b>Istruzioni per l'uso/Descrizione tecnica</b><br>Pinzette                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
| pt  | <b>Instruções de utilização/Descrição técnica</b><br>Pinças                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
| nl  | <b>Gebruiksaanwijzing/Technische beschrijving</b><br>Pincetten                                                                                                                                                                                                                                                                                                                                                                                                                                                                       |
| da  | <b>Brugsanvisning/Teknisk beskrivelse</b><br>Pincetter                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| sv  | <b>Bruksanvisning/Teknisk beskrivning</b><br>Pincetter                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| fi  | <b>Käyttöohje/Tekninen kuvaus</b><br>Atulat                                                                                                                                                                                                                                                                                                                                                                                                                                                                                          |
| lv  | <b>Lietošanas instrukcijas/tehniskais apraksts</b><br>Pincetes                                                                                                                                                                                                                                                                                                                                                                                                                                                                       |
| lt  | <b>Naudojimo instrukcija/techninis aprašas</b><br>Pincetai                                                                                                                                                                                                                                                                                                                                                                                                                                                                           |
| ru  | <b>Инструкция по применению/Техническое описание</b><br>Пинцеты                                                                                                                                                                                                                                                                                                                                                                                                                                                                      |
| cs  | <b>Návod k použití/Technický popis</b><br>Pinzety                                                                                                                                                                                                                                                                                                                                                                                                                                                                                    |
| pl  | <b>Instrukcja użytkowania/Opis techniczny</b><br>Pincety                                                                                                                                                                                                                                                                                                                                                                                                                                                                             |
| sk  | <b>Návod na použitie/Technický opis</b><br>Pinzety                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |
| hu  | <b>Használati útmutató/Műszaki leírás</b><br>Csipeszek                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| sl  | <b>Navodila za uporabo/Tehnični opis</b><br>Pincete                                                                                                                                                                                                                                                                                                                                                                                                                                                                                  |
| hr  | <b>Upute za uporabu/Tehnički opis</b><br>Pincete                                                                                                                                                                                                                                                                                                                                                                                                                                                                                     |
| ro  | <b>Manual de utilizare/Descriere tehnică</b><br>Pensetă                                                                                                                                                                                                                                                                                                                                                                                                                                                                              |
| bg  | <b>Упътване за употреба/Техническо описание</b><br>Пинсети                                                                                                                                                                                                                                                                                                                                                                                                                                                                           |
| tr  | <b>Kullanım Kılavuzu/Teknik açıklama</b><br>Forsepsler                                                                                                                                                                                                                                                                                                                                                                                                                                                                               |
| el  | <b>Οδηγίες χρήσης/Τεχνική περιγραφή</b><br>Λαβίδες                                                                                                                                                                                                                                                                                                                                                                                                                                                                                   |

Aesculap AG | Am Aesculap-Platz | 78532 Tuttlingen | Germany  
Phone +49 (0) 7461 95-0 | Fax +49 (0) 7461 95-26 00 | [www.aesculap.com](http://www.aesculap.com)

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