COMFYMASK - COMFYMASK FIT**SEMI-MASCHERA FILTRANTE MONOUSO FFP2 NR**

ENGLISH

FFP2 NR FILTERING HALF MASK

FRANÇAIS

DEMI-MASQUE FILTRANT JETABLE FFP2 NR

ESPAÑOL

SEMI-MASCARILLA FILTRANTE DESECHABLE FFP2 NR

PORTUGUÊS

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FFP2 NR FILTERING HALF MASK

Compliant with Reg. EU 2016/425 - Class FFP2 as per EN 149:2001+A1:2009 Filtration efficiency greater than 94%.

PERSONAL PROTECTIVE EQUIPMENT - APPLICATIONS

Protection against non-toxic aerosols and airborne particles.
Not suitable for protection against toxic gases and vapours, epoxy fumes. Not suitable for use in case of fire.

WARNINGS AND DIRECTIONS FOR USE

Failure to comply with the directions for use and warnings regarding use of the product may reduce the efficiency of the face mask and cause disease and death. Use after receiving adequate training.
The user must ensure the face mask is fit for the intended use.
Do not use this face mask in atmospheres with an oxygen concentration less than 17%.
Do not use in areas where contaminant concentrations pose an immediate hazard to health. Do not use in explosion risk areas.
Immediately abandon the work area in case of difficulty breathing or dizziness.
The presence of a beard may drastically reduce the efficiency characteristics of the face mask. The face mask is disposable, it must not be used for more than one work shift.
After use, dispose of the product as general waste.
Keep unused face masks in their original, intact packaging.

CHECKS BEFORE USE

Check the integrity of the face mask before use. Do not modify or alter the face mask. Do not use and discard the face mask if torn or stained.

INSTRUCTIONS FOR USE

1. Pick up the face mask with the outer side facing down. Hold the elastics with your fingers.
2. Position the face mask under your chin, with the nose piece facing upwards.
3. Hook the elastics around your ears.
4. Using two fingers, gently press on the nose piece band so that it adheres to your nose.
5. USER SEAL CHECK: place both hands over the mask and breathe out sharply, if air leaks out between the mask and your face, re-adjust the device and nose piece. Repeat the seal check

Valid for 3 years

Scan the QR code to download the declaration of conformity
Store at a temperature between -20°C and +40°C and humidity <80%
1 piece per pack

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