CLOSE FORMAT

75 mm

52 mm

OPEN FORMAT

75 mm

75 mm

74 mm

CONDOMS • INSTRUCTIONS FOR USE

- Before opening, check the expiry date and make sure that the packaging is not damaged. Never use a condom if it has expired or the packaging is broken.
- Do not use a condom if the material is sticky, brittle or visibly damaged.
- Open the packaging carefully, pulling at the serrated side, and handle the condom carefully. Do not use scissors or other sharp objects. Condoms may be damaged by nails or sharp objects, such as jewellery, zips or buckles.
- Remove the condom carefully from its packaging and ensure that the part to unroll is on the outside. If it is inside, the condom is the wrong way round. The condom must be placed on the erect penis before any contact between it and the body of the partner due to frequent pre-ejaculatory secretions of sperm, which may result in possible unintended pregnancy and/or sexually transmitted diseases.
- Gently squeeze the end of the condom tip between the thumb and forefinger to prevent any air from being trapped inside which may cause breakage.
- Continuing to hold the tip of the condom, position the condom on the glans of the erect penis unrolling it carefully over the entire length up to the base of the penis using the other hand. If the condom rolls up during intercourse, roll it down immediately. If the condom slips off, put on a new condom before intercourse.
- Immediately after ejaculation, slowly pull out the penis while still erect keeping the condom tight at the base of the penis, so that it does not slip off.
- Then remove the condom by sliding it off slowly. After this is done it is important that the penis and the condom do not come back in contact with the partner.
- Tie a knot at the base of the condom to prevent any spillage of semen. Dispose of the condom by wrapping it in paper and throwing it in a bin; do not throw condoms down the toilet.

IMPORTANT POINTS TO KNOW BEFORE USING A CONDOM

- DISPOSABLE. DO NOT REUSE THE SAME CONDOM AFTER USE.** Condoms are disposable. You must use a condom for every sexual intercourse.
- Keep in a cool dry place away from heat or direct exposure to sunlight. Heat can degrade the physical properties of condoms.
- If you need additional lubrication use water-based lubricants.
- Avoid using oil-based lubricants such as Vaseline, mineral oil, paraffin, butter, body lotions, massage oils and vaginal suppositories, because they weaken the condom and reduce functionality of the product.
- It is necessary to consult a doctor or pharmacist about the compatibility between condoms and prescription or over the counter drugs that are put on the penis or vagina.
- If at the end of sexual intercourse, for whatever reason, there was reason to fear an unwanted pregnancy or to have been infected, seek immediate medical attention within 72 hours.
- In the case of anal intercourse it is advisable to further lubricate the exterior of the product to reduce the risk that the same will slip off or break.
- Designed for vaginal use.

CAUTION

The product may elicit anaphylactic responses in individuals who are allergic to latex. Avoid contact with eyes and do not use in case of rash or sores on the skin. In case of allergic reactions it is recommended to discontinue use and consult a doctor. If during use you ever experience breathing and circulation (cyanotic lips, etc.) difficulties, immediately discontinue use and consult a doctor. Although no method of contraception can provide 100% protection, when used properly it can protect from unwanted pregnancies and sexually transmitted diseases such as HIV (AIDS), Chlamydia, genital herpes, genital warts, hepatitis B and syphilis. The user is required to notify the Manufacturer and the competent Authorities of serious accidents that have occurred with the medical device.

| | | | |
|-------------------------------|-------------|--|-------------|
| SIRE <i>Classic</i> | EN ISO 4074 | CE  EUROSIREL S.p.A. Viale Europa, 30 0373 20047 Cusago - (MI) ITALY | EN 10 22011 |
|-------------------------------|-------------|--|-------------|

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