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**FASTENING ELASTIC BELTS  
FASTENING ELASTIC RUBBER BELTS  
ATRAUMATIC BUTTON  
DIRECTIONS FOR USE**

**DESCRIPTION AND INDICATION**

**FASTENING ELASTIC BELTS**

Electrode fastening elastic belts with Velcro on both ends.

**FASTENING ELASTIC RUBBER BELTS**

Electrode fastening rubber belts with holes. The fastening of these belts is ensured by a non-traumatic fastening button.

Warning: this product contains natural latex rubber that may cause anaphylactic reactions in allergic subjects.

**USE**

**FASTENING ELASTIC BELTS**

Place electrode on selected application area. Wrap fastening belt around electrode and fix Velcro. If necessary, male Velcro may be fastened on any point of the belt. Male Velcro won't stick until belt is properly tightened.

**FASTENING ELASTIC RUBBER BELTS WITH NON-TRAUMATIC FASTENING BUTTON**

Place electrode on selected application area and fasten it with the rubber belt. Insert button support into a hole and wrap fastening belt around application area. Fasten belt to the hole ensuring proper tightening.

**WARNINGS**

- Non-sterile product.
- Electrostimulation and ECG units, electrodes and connection cables to be used with present product shall be in accordance with regulations in force.
- Always refer to instructions supplied by the producer for proper use of all units and electrodes.
- Medical devices should be connected and activated by qualified personnel only.
- Don't use belts if patient's skin in selected area is scraped, injured or reddened.
- Don't stretch belts too tight to prevent vein obstruction or belt ripping.
- The products must be used by qualified healthcare personnel.

**CLEANING**

For cleaning of elastic belts hand washing with neutral soap or fine cloth programmed machine washing in cold water is suggested.

For cleaning of rubber belts it is suggested to use normal cleaning and disinfecting detergents. After cleaning rinse under cold running water. After drying, belts should be sprinkled with talc to prolong their life. For use of any detergent and disinfectant always refer to instructions supplied.

**LIFE OF PRODUCT**

- Elastic belts: permanent if belt is properly washed and fastened by sticking together male and female Velcro. In case of loss of elastic properties, the product must be disposed of.
- Rubber belts: permanent. In case of visible defects and/or loss of elastic properties, the product must be disposed of.

**STORAGE**

The product must be stored in the original package at the environmental conditions (temperature and relative humidity) specified on the pouch's label. Putting external heavy weights on the package, the product could be damaged.

**WASTE DISPOSAL**

Refuses deriving from health structures must be disposed in according to the regulation in force.

**GENERAL NOTES**

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. For any malfunction or defect of the device, inform the Manufacturer's Quality Service.

**CAUTION: U.S. FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A PHYSICIAN**

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