KLY® LUBRICATING JELLY (DE)

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MARKA/KLY

KLY® LUBRICATING JELLY (EN)

PRODUCT NAME: KLY® LUBRICATING JELLY
COMPOSITION: Deionized water, Natural Glycerin, MonopropylenGlycol, Hydroxy Ethyl Cellulose, Methyl Hydroxybenzoate, Citric Acid
INTENDED USE: This gel is used in endoscopy, colonoscopy, gynecological examinations, rectal endoscopic, catheter operations, also used for vaginal lubrication.
INSTRUCTION FOR USE: To Open The Tube: Remove and reverse the cap then puncture the tube with it. Apply Lubricating Jelly to the gloves or the instrument in sufficient amount so that it forms a film over the area that comes in contact with patient. Wash off after use.
To Open The Sachet: Open carefully from marked area.
SPECIAL PRECAUTIONS: Precautions: If container is damaged or opened, the Product is not guaranteed.
CONTRAINDICATION: There is no known contraindication
OVERDOSAGE AND SIDE EFFECTS: There is no known symptoms for overdose usage.
SENSITIVITY TO GLYCERIN: Glycerin has emollient properties. Precautions Shall be taken when using the gel in people with sensitivity to glycerin.
IDENTIFICATION: Transparent, water-soluble, greasless and jelly.
PRESENTATION: Tubes Containing either: 42 g, 50 g, 82 g, 113,4 g, 100 g or 250 g
SACHETS CONTAINING EITHER: 2,5 g, 3 g, 3,5 g, 4 g, 4,5 g, 5 g, 10 g
STORAGE INSTRUCTIONS: Store between 5-30°C
KEEP OUT OF REACH OF CHILDREN

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