



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. - Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel. 199 400 401 - fax 199 400 403
Export: tel. +39 02 953854209/221/225 - fax +39 02 95380056
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

OROLOGIO DIGITALE PER INFERMIERI DIGITAL NURSE WATCH MONTRE D'INFIRMIÈRE DIGITALE RELOJ DIGITAL PARA ENFERMERA

Manuale d'uso - User manual
Manuel de l'utilisateur - Guía de uso



ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.
ATTENTION: The operators must carefully read and completely understand the present manual before using the product.
AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.
ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

REF 20480



Gima S.p.A.
Gessate (MI) - Italy
Made in China





Functions

6 digits display: Month, Date, Week day, Hour, Minute, Second
 Stop watch
 12/24 hours
 Alarm
 EL background light



Instructions

Set Time & Date, week day

3 x Press B, second flashes, set time by pressing D,C, by pressing C to confirm & return normal display. Please set time by following steps.

3xPress B (Example: **13:02:23**)

Second (23) flashes, press D to increase the seconds, press C back to 00, press C to confirm.

Minute (02) flashes, press D to increase the numbers, press C to confirm.

Hour (13) flashes, press D to increase the numbers, press C to confirm.

Date flashes, press D to increase the numbers, press C to confirm

Month flashes, press D to increase the numbers, press C to confirm

Week flashes, press D move the flash to the correct week day, then press B to confirm all setups, and back to normal display.



Stop watch

Under normal display, 1 x press B
enter stopwatch mode,
1 x press D Start/stop to counting time
1 x press C clear data.
1 x press B, back to normal display.

12/24 Hours

Under normal display , 3 x press B,
enter time setup mode,
2 x press C enter 12/24 switch mode,
press D until 12/24 displays.
1 x press B back to normal display.



EL background light

1 x press A, EL light on 3 seconds.

Set Alarm

Under normal display.
2 x press B, enter alarm mode, hour flashes, press D to adjust the hour, then
press C, minute flashes, press D to adjust minute, press B back to normal
display.

Alarm On/Off

Press C(don't remove), press D at the same time to set ON/OFF of alarm.
Alarm keeps on 1 minute.

Hourly bell

Press C(don't remove),press B to set hourly bell on/off. When hourly bell is
on, all week day display in on.
Otherwise means hourly bell is off.

Battery replacement

On the back of the watch, there is a notch to open the housing and get to
the battery. Lift the rear with a sharp and thin instrument. Remove the metal
cover and the battery by sliding it on the side.
Insert the new battery complying with the correct polarity with the "+" facing
up. Replace the metal cover and close the case.

Weight: 45g

Battery: 3V Lithium Battery CR2025



Disposal: *The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.*

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed.

The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.