



GIMA

PROFESSIONAL MEDICAL PRODUCTS

FORBICI PER BENDE BANDAGE SCISSORS CISEAUX À BANDAGES TIJERAS PARA VENDAS VERBANDSCHERE TESOURAS PARA LIGADURAS

REF

CR-SC01 4.7" (GIMA 20560)
CR-SC01 6.5" (GIMA 20570)
CR-SC01 6.5" (GIMA 20571)
CR-SC01 6.5" (GIMA 20572)
CR-SC01 6.5" (GIMA 20573)
CR-SC01 6.5" (GIMA 20574)
CR-SC01 7.5" (GIMA 20580)
CR-SC01 7.5" (GIMA 20581)
CR-SC01 7.5" (GIMA 20582)
CR-SC01 7.5" (GIMA 20583)
CR-SC01 7.5" (GIMA 20584)



WUXI EMSRUN TECHNOLOGY CO.,LTD
No.100 Fengbin Road,WUXI city, JIANGSU
province, China - Made in China



SUNGO Europe B.V.,
Olympisch Stadion 24,
1076DE Amsterdam, Netherlands



Gima S.p.A.
Via Marconi, 1 - 20060
Gessate (MI) Italy
gima@gimaitaly.com
export@gimaitaly.com
www.gimaitaly.com
Made in China



ENGLISH

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

The blades of bandage scissors are made of 2Cr13 stainless steel, the handles of PP.
The blades of these scissors have been milled in order to cut clothes, gauzes and bandages.

Bandage scissors are suitable for:

- Cutting medical gauze
- Cutting bandages away from skin

Warning

Remember that scissors may cause harm if not handled with care.

Please pay due attention and care during use.

Choose a safe place for storing the scissors, out of reach of non-authorised personnel.




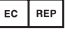

Clean the scissors after use and eliminate any residue.

Keep the scissors closed when not in use. Any equipment showing signs of wear should be discarded immediately.

Instructions for use









Firmly grip the scissors, cut starting from the front of the body. Do not cut laterally, pay attention in order not to damage the patient's skin.

Sterilisation: autoclave at 138°C

	Caution: read instructions (warnings) carefully
	Medical Device
	Date of manufacture
	Authorized representative in the European community
	Imported by

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

	Keep in a cool, dry place
	Keep away from sunlight
	Consult instructions for use
	Product code
	Lot number
	Non-sterile
	Manufacturer
	Medical Device compliant with Regulation (EU) 2017/745