180.00 mm

Value&Safe

Particle Filtering Mask - Earloop - white
 Masque filtrant contre les particules - Avec élastique - blanc
 Mascarilla filtrante contra partículas - Con elástico - blanca
 Máscara de filtragem de partículas - Com elástico - branca
 Partikelfiltermaske - Mit elastischem Ohrband - weiß

FFP2 NR

1
 Piece
 Pièce
 Pieza
 Peça
 Stück

 It should be worn in order to block the exhalation or ejection of contaminating particles from the mouth and from the nasal cavity into common spaces - **NON-STERILE, SINGLE-USE, DUSTPROOF** - 42,8% **nonwoven fabric**, 28,6% **hot-air cotton**, 28,6% **meltblown fabric** - Do not press excessively during transport, in order to avoid damaging the package. 3-year validity, do not use beyond expiration date.

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EU: 2016/425 - EN 149:2001 + A1:2009

Wearing instructions

- Place the nose piece on your nose and the mask on your chin.
- Pull the elastic loops behind your ears and adjust the mask.
- Adjust the nose piece to your nose.
- Make sure that the mask fits close to your face.

WARNING

- a. Risk may be expected if the product is used in a surgical setting or in a setting with significant exposure to bodily or other hazardous liquids or fluids; or used in a clinical setting where the risk of infection through inhalation is high; or used in the presence of a high intensity heat source or flammable gas.
- b. The product is not designed for use in an anoxic or an underwater environment.
- c. Check for integrity before use.
- d. This product is not a respirator.
- e. The mask does not eliminate the risk of contracting any disease or infection.
- f. Failure to properly use and maintain this product could result in illness.
- g. Do not use beyond expiration date.
- h. Do not use if the package is damaged.
- i. Do not use with oxygen and use only in ventilated areas with oxygen >19% and only for a single work shift. The device is less effective if it is altered or damaged or if hair and beard are present. If you feel sick, leave the working area and move to an aerated area. Do not use in highly contaminated or potentially explosive areas.

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 Anhui Lekang Sanitary Materials Co., Ltd.
 Qingcao Industrial Park, Tongcheng,
 Anqing City, Anhui Province
 Made in China

Imported by / Importé par / Importado por
 Importado por / Eingeführt von:
 Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com
 www.gimaitaly.com




**LOT**

EN 149:2001 + A1:2009

Notified body / Organisme notifié
 Organismo notificador
 Organismo notificado
 Benannte Stelle
 0370

LGAI Technological Center (APPLUS)
 Campus UAB,
 Ronda de la Font del Carme s/n,
 E-08193, Bellaterra (Barcelona), Spain

Regulation EU 2016/425
 Règlement UE 2016/425
 Regulamento UE 2016/425
 Regulamento da UE 425/2016
 Verordnung EU 2016/425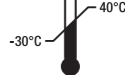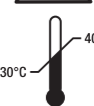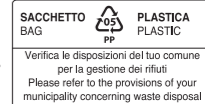

Rev. 5 - 12/21