509520 Rev. 2 - 09/20
Size 14,5x13,5x9cm

Universal Certification
NB 2163
Necip Fazil Bulvarı Keşap Sitesi E2
Blok No:4/84 Yukarı Dudullu Ümraniye
İSTANBUL / TURKEY

Imported by / Importiert von / Importado por / Importado por:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

300003

REF 20740
Model / Modèle / Modell
Modelo / Modelo
TF-9006
Tengtel Technology Co., Ltd.
Address: No. 11 Tongqiu Road,
Zhangpu Town, Kunshan City, Jiangsu Province
Made in China

Instructions - Instructions - Bedienungsanleitung Instrucciones - Instruções

Category: disposable, no air valve, ear wearing
Catégorie: jetable, sans soupape, avec élastique
Kategorie: zum Einmalgebrauch, ohne Ventil, mit elastischem Ohrband
Categoría: desechable, sin válvula, con elástico
Categoria: descartável, sem válvula, com elástico

Spec.: 20 pcs, average size / **Spéc.:** 20 pièces, taille moyenne
Spez.: 20 Stücke, Mittelgröße / **Espec.:** 20 piezas, tamaño mediano
Espec.: 20 unidades, tamanho médio

Material: non-woven fabric, Hot-Air Cotton, meltblown
Matériau: tissu non tissé, coton d'air chaud, tissu soufflé par fusion
Werkstoff: Vliesstoff, Heißluftbaumwolle, schmelzgeblasener Stoff
Material: tejido-no-tejido, algodón de aire caliente, tejido soplado fundido
Material: tecido não-tecido, algodão de ar quente, tecido fundido

LOT

Execution Standard / Normes d'exécution / Durchführungsstandards
Normas de ejecución / Normas de fabrico:
EN149:2001+A1:2009 FFP2 NR - UE 2016/425

Storage condition: avoid direct sunlight, humidity <80%, no corrosive gas, store in the clean room with good ventilation.

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20 Pieces
Pièces
Stücke
Piezas
Peças

FFP2 NR

Matters needing attention and warnings

- Check the appearance before using, please do not use it when it is damaged or soiled.
- Do not wash, heat or disinfect the mask.
- Do not touch the inside layer of the mask when taking off to avoid pollution.
- Do not use when there is not enough oxygen or in poisonous gas.
- Change the mask when it has been damaged or has undesirable odor.
- Not suitable for: 1 Individuals with a compromised respiratory system, such as asthma or emphysema. 2 Elders or patients who cannot express their body condition or control their hands freely.

Warnings
1. Wearing the mask incorrectly will fail to achieve the protective effect.
2. Take off it when you feel sick or there is strong irritating odor.

Wearing instruction
Wear the mask according to the instructions below to achieve the best protective effect.

- Take out the mask, do not touch the inside layer.
- Unfold the mask and put it on your face.
- Adjust the strap on both sides of the mask to your ears.
- Adjust the nasal splint position.

Applications/restrictions: personal protective equipment for the effective protection of the respiratory tract of workers exposed to dusts, fumes, fogs and potential irritants of the throat and of the upper airways.

Regulation (EU) 2016/425

Risks: no protection against particles of radioactive substances, nor against airborne biological substances from risk group 3 and against enzymes.

The particle filtering mask is intended for use for a single work shift (NR).

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