

**MASCHERINA CHIRURGICA LAVABILE DI TIPO II DOPPIO STRATO AD USO MEDICALE**  
**TYPE II WASHABLE DOUBLE LAYERED SURGICAL MASK FOR MEDICAL USE**  
**MASQUE CHIRURGICAL LAVABLE À DOUBLE COUCHE DE TYPE II À USAGE MÉDICAL**

**TYP II WASCHBARE DOPPELLAGIGE CHIRURGISCHE MASKE FÜR MEDIZINISCHE ANWENDUNGEN**

**MASCARILLA FACIAL QUIRÚRGICA DE DOS CAPAS TIPO II LAVABLE PARA USO MÉDICO**  
**MÁSCARA CIRÚRGICA DE CAMADA DUPLA LAVÁVEL TIPO II PARA USO MÉDICO**

**TYP II TVÄTTBART KIRURGMUNSKYDD MED DUBBELT SKIKT FÖR MEDICINSK ANVÄNDNING**


**ΠΛΕΝΟΜΕΝΗ ΧΕΙΡΟΥΡΓΙΚΗ ΜΑΣΚΑ ΔΙΠΛΗΣ ΣΤΡΩΣΗΣ ΤΥΠΟΥ II ΓΙΑ ΙΑΤΡΙΚΗ ΧΡΗΣΗ**



**DWUWARSTWOWA MASKA CHIRURGICZNA MEDYCZNA MOŻLIWA DO PRANIA TYPU II**  
**MASCĂ CHIRURGICALĂ DE UZ MEDICAL, DE TIP II, LAVABILĂ, CU 2 STRATURI**

قناع جراحي بطبقة مزدوجة قابل للغسيل ومن النوع الثاني للاستخدام الطبي

Distribuito da / Distributed by / Distribué par / Vertrieben von  
 Distribuido por / Distribuído por / Distribuemas av / Διανέμεται από / Rozprowadzana przez / Distribuit de / توزیع :



**REF 20750 / BP2020V3TN FS** 

 MYCROCLEAN ITALIA SRL  
 Via Carlo Porta, 14  
 20064 Gorgonzola - Italy  
 Made in Italy  

**LOT** Vedere etichetta / See label / Voir l'étiquette  
 Siehe Etikett / Ver la etiqueta / Ver rótulo  
 Se etikett / Δείτε ετικέτα / Patrz etykieta  
 Consultăți eticheta / انظر الملصق

Vedere etichetta / See label / Voir l'étiquette  
 Siehe Etikett / Ver la etiqueta / Ver rótulo  
 Se etikett / Δείτε ετικέτα / Patrz etykieta  
 Consultăți eticheta / انظر الملصق

UNI EN 14683:2019



000001



**IT** - La mascherina deve essere indossata come mostrato nella foto. Parte piccola sul naso, parte grande sotto

**GB** - The mask must be worn as shown in the photo  
 Small part on the nose, large part below.

**FR** - Le masque doit être porté de la façon montrée dans la photo : petite partie sur le nez, grande partie en-dessous.

**DE** - Die Maske muss, wie in der Abbildung gezeigt, getragen werden, d.h. der kleinere auf der Nase und der größere Teil unten.

**ES** - La mascarilla se debe colocar tal y como muestra la foto. Parte pequeña en la nariz, parte grande abajo.

**PT** - A máscara deve ser utilizada conforme mostrado na foto. Parte pequena sobre o nariz, parte grande para baixo.

**SE** - Ansiktsmasken ska bäras som fotot visar. Mindre del på näsan, större nedtill.





**GR** - Η μάσκα πρέπει να φοριέται όπως φαίνεται στη φωτογραφία. Το μικρό τμήμα στη μύτη, το μεγάλο τμήμα.

**PL** - Maseczka musi być noszona w sposób pokazany na zdjęciu stronie. Mniejsza część na nosie, większa część pod brodą.

**RO** - Masca trebuie purtată așa cum se observă în fotografia. Partea mai îngustă pe nas, partea mai lată dedesubt.

**SA** - يجب ارتداء القناع كما هو موضح في الصورة على اليمين، الجزء الصغير على الأنف، الجزء الكبير أدناه

**ENGLISH**

 REF	Product code	 LOT	Lot number
 Manufacturer			Date of manufacture
 Non-sterile			Consult instructions for use
	Medical Device complies with Directive 93/42/EEC		

**1 - Product description and intended use**  
 Washable mask, in 99% polyester and 1% carbon fibre fabric, designed to protect the nose and mouth.

**2 - Classification**  
 The BP2020V3TN series masks are classified as Type II, according to the UNI EN 14683:2019 standard.

**3 - Features**  
 The device consists of a 99% polyester and 1% carbon fibre fabric layer and of a water-repellent outer layer; it is machine washable up to 60°C (the fabric withstands up to 150°C) It is not sterile.

**PFE values: Particle Filtration efficiency (%) 0.3 µm 0.5 µm at 95%.**

The fabric is **NON-CYTOTOXIC** according to tests carried out on fibroblasts in compliance with the UNI EN ISO 10993-5 standard.

**BFE Filtration**  
 BFE = 99.8% (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard).

**Biological load**  
 UFC <30 UFC (from laboratory tests carried out in compliance with the UNI EN 14683:2019 standard).

**Air permeability (cm<sup>2</sup>/cm<sup>3</sup>/sec) ASTM E96** (Also known as ISO - 11092 or Ret or Hohenstein test).  
 The result obtained on the BP2020 series masks is 7.5, therefore good or very breathable.

**4 - Instructions for use and warnings**  
 Handle with clean and sanitised hands, making sure the mask adheres well to the face, in the right position, i.e. with the smallest part towards the nose, and the larger one over the mouth and chin.  
**DO NOT MODIFY THE MASK; DO NOT CUT THE ELASTIC BANDS; WEAR IT RIGHT WAY OUT** (as shown in the figure).  
**Otherwise the mask loses its protective function.**

**5 - Washing**  
 By hand or in the washing machine, with a detergent, up to 60°C. **DO NOT BLEACH.**  
 By complying with the instructions for use and maintenance, the product maintains the same physical and technical features up to approx. 100 washes.

**6 - Disposal**  
 Mask to be disposed of as SUW (unless it has been contaminated, in which case it must be disposed of as "special waste"). The outer bag is disposed of as plastic.