



PROFESSIONAL MEDICAL PRODUCTS

## DISPOSABLE STANDARD SMMS GOWN

## ENGLISH

**Product Name** Disposable Standard SMMS Gown  
**Product Code** FGS-SG-SM50-L, FGS-SG-SM50-XL,  
 FGS-SG-SM50-XXL  
**Product Size** Unit: cm

GIMA CODE	21046	21047	21048	21049
FULL GUARD CODE	FGS-SG-SM50-M	FGS-SG-SM50-L	FGS-SG-SM50-XL	FGS-SG-SM50-XXL
Sizes	120x140 (M)	120x150 (L)	130x150 (XL)	140x160 (XXL)
Length (Shoulder to hem)	120	120	130	140
Width (monolayer)	140	150	150	160
Belt length	65	65	65	69
Sleeve Length	58	59	59	61
Deviation	±2	±2	±2	±2

**Product Model** Gown Type. M/L/XL/XXL

**Structure and Composition** The product uses 50g SMMS non-woven fabric. It is composed of front, back, sleeve, ties and cuff. The technology process is by cutting, ultrasonic welding, ties, velcro, cuff, folding, sealing.

**Intended use** These items are sterilized by EO method and they are used as a barrier to liquid and microbial penetration between patient and operation room, other surgical devices and the operation room personnel.

**Contraindication** The products can be used only once and should be used immediately after opening the package. If

the package is damaged, the product is wet or moldy. Don't use them.

**How to use the product**

Open the package and take out the product, gently shake the gown, lift the sides of the collar, insert the hands into the sleeves, extend the arms forward and tie the belts.

**Storage** It should be stored in a clean room with relative humidity not exceeding 80%, no corrosive gas, stored in a cool, dry and well ventilated.

**Warning**

1. This product is disposal product, don't reuse.
2. This product is single use, never share your product with another.
3. It is forbidden to use when the packaging or the products are damaged.

**Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user or patient is established.**





















**Validity** 3 years.

**Date of production** See date on packaging

**Date of expiration** See date on packaging


**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

 REF	Product code		Keep away from sunlight		Keep in a cool, dry place
 LOT	Lot number		Medical Device compliant with Regulation (EU) 2017/745		Caution: read instructions (warnings) carefully
	Manufacturer		Date of manufacture		Medical Device
 STERILE EO	Sterilized using ethylene oxide		Expiration date		Don't use if package is damaged
	Humidity limit		Consult instructions for use		Disposable device, do not re-use
	Temperature limit		Authorized representative in the European community		Fragile, handle with care
	This side up		Unique device identifier		

**GIMA 21046**  
**GIMA 21047**  
**GIMA 21048**  
**GIMA 21049**

 Xianning Full Guard Medical Products Co., Ltd  
 Yongan East Road Wenquan Economic Development Zone, Xianning City, Hubei Province, China  
 Made in China

 **FGS-SG-SM50-M**  
**FGS-SG-SM50-L**  
**FGS-SG-SM50-XL**  
**FGS-SG-SM50-XXL**

 **ZOUSTECH S.L.**  
 Pso. Castellana, 141 - Planta 19,  
 28046 - Madrid, Spain

Importato da / Imported by:

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

[www.gimaitaly.com](http://www.gimaitaly.com)

