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GB

Product Name Disposable Cap With Ties
Product Code FGDCT-SM35 (21098)
Product Model Ties on cap, 62x14.5cm
Structure and Composition The cap is made of 35-40gSMS, with ties for fixing.
Intended use The products are mainly used for general isolation in outpatient clinics, wards, inspection rooms, cleaning room, laboratory etc. It is intended to be used by healthcare professionals in hospitals and clinics to maintain a good level of hygiene and contribute to infection control.
Contraindication The products can be used only once and should be used immediately after opening the package. If the package is damaged, the product is wet or moldy. Don't use them.
How to use the product
1. Open the product package and take out of the cap, and squeeze the ends of the cap with both hand.
2. Raise your hand and put the cap on your head and adjust it to make sure your hair is not leaking.
Storage It should be stored in a clean room with relative humidity not exceeding 80%, no corrosive gas, stored in a cool, dry and well ventilated.

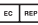
- **Warning**
1. This product is disposal product, don't reuse.

2. This product is single use, never share your product with another.

3. It is forbidden to use when the packaging or the caps are damaged.

Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user or patient is established.

Validity 3 years.
Date of production See date on packaging
Date of expiration See date on packaging

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GIMA 21098

Xianning Full Guard Medical Products Co., Ltd
Yongan East Avenue, Xianning Economic Development
Zone, Xianning City, Hubei Province, China
Made in China

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GmbH Gneisenaustraße 8.40477,
Gima S.p.A.