



PROFESSIONAL MEDICAL PRODUCTS

DISPOSABLE ISOLATION GOWN

ENGLISH

Product Name Disposable Isolation Gown
Product Code FGIS-PP25-M, FGIS-PP25-L
Product Size Unit: cm

GIMA CODE	21131	21132
FULL GUARD CODE	FGIS-PP25-M	FGIS-PP25-L
Sizes	115x137 (M)	120x150 (L)
Length (Shoulder to hem)	115	120
Width (monolayer)	137	150
Belt length	180	180
Sleeve Length	58	59
Deviation	±2	±2

Product Model Gown Type. M/L

Structure and Composition The product uses 25g PP non-woven fabric. It is composed of front, back, sleeve, ties and cuff. The technology process is by cutting, sewing, ties, cuff, folding, packing.

Intended use The products are mainly used for general isolation in outpatient clinics, wards, inspection rooms, cleaning room, laboratory etc. It is intended to be used by healthcare professionals in hospitals and clinics to maintain a good level of hygiene and contribute to infection control.

Contraindication The products can be used only once and should be used immediately after opening the package. If the package is damaged, the product is wet or moldy. Don't use them.

How to use the product

Open the package and take out the product, gently shake the gown, lift the sides of the collar, insert the hands into the sleeves, extend the arms forward and tie the belts.

Storage It should be stored in a clean room with relative humidity not exceeding 80%, no corrosive gas, stored in a cool, dry and well ventilated.

Warning

1. This product is disposal product, don't reuse.
2. This product is single use, never share your product with another.
3. It is forbidden to use when the packaging or the products are damaged.

Notice: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and competent authority of the Member State in which the user or patient is established.

















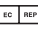



Validity 3 years.


Date of production See date on packaging


Date of expiration See date on packaging


GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

 REF	Product code		Keep away from sunlight		Keep in a cool, dry place
 LOT	Lot number		Medical Device compliant with Regulation (EU) 2017/745		Caution: read instructions (warnings) carefully
	Manufacturer		Date of manufacture		Medical Device
	Non-sterile		Expiration date		Don't use if package is damaged
	Humidity limit		Consult instructions for use		Disposable device, do not re-use
	Temperature limit		Authorized representative in the European community		Fragile, handle with care
	This side up		Handle with care		

 **FGIS-PP25-M (GIMA 21131)**
FGIS-PP25-L (GIMA 21132)

 Xianning Full Guard Medical Products Co.,Ltd
 Yongan East Avenue,
 Xianning Economic Development
 Zone, Xianning City, Hubei Province, China
 Made in China

 Humiss Beratung
 GmbHGneisenastraße
 8.40477, Düsseldorf, Deutschland

Importato da / Imported by:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
 gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

