**CATETERE URETRALE STERILE MONOUSO**  
**STERILE URETHRAL CATHETER FOR SINGLE USE**  
**CATHÉTERA URÉTRAL STÉRILE À USAGE UNIQUE**  
**CATÉTER URETRAL ESTÉRIL DE UN SOLO USO**  
**CATETER URETRAL ESTÉRIL DE UTILIZAÇÃO ÚNICA**  
**STERILER HARNRÖHRENKATHETER FÜR DEN EINMALGEBRAUCH**  
**ΑΠΟΣΤΕΙΡΩΜΕΝΟΣ ΟΥΡΗΘΡΙΚΟΣ ΚΑΘΗΤΗΡΑΣ ΜΙΑΣ ΧΡΗΣΗΣ**  
**STERYLNY CEWNIK UROLOGICZNY DO JEDNORAZOWEGO UŻYTKU**  
**STERILNÍ URETRÁLNÍ KATÉTR NA JEDNO POUŽITÍ**  
**STERIL URINRÖRSKATETER FÖR ENGÅNGSBRUK**  
**STERIILI JA KERTAKÄYTTÖINEN VIRTSAPUTKIKATETRI**  
**STERILNI URETRALNI KATETER ZA ENKRATNO UPORABO**  
**STERILNÝ MOČOVÝ KATÉTER NA JEDNO POUŽITIE**  
**CATETER URETRAL STERIL DE UNICĂ FOLOSINȚĂ**  
**STERIELE URETHRALE KATHETER VOOR EENMALIG GEBRUIK**  
**STERILNI KATETER ZA URETRU ZA JEDNOKRATNU UPORABU**  
**STERIL URINRÖRSKATETER FÖR ENGÅNGSBRUK**  
**STERILT URINRØRSKATETER TIL ENGANGSBRUG**  
**СТЕРИЛЕН УРЕТРАЛЕН КАТЕТЪР ЗА ЕДНОКРАТНА УПОТРЕБА**  
**VIENKARTINIO NAUDOJIMO STERILUS ŠLAPLĖS KATETERIS**  
**STERILS URĪNIZVADKANĀLA KATETRS VIENREIZĒJAI LIETOŠANAI**  
**STERIILNE URETRA KATEETER ÜHEKORDSEKS KASUTAMISEKS**

**REF** S5122.12 (Gima 22054) - S5122.14 (Gima 22055) - S5122.16 (Gima 22056) - S5122.18 (Gima 22057) - S5122.20 (Gima 22058) - S5122.22 (Gima 22059) - S5122.12 (Gima 22064) - S5122.14 (Gima 22065) - S5122.16 (Gima 22066) - S5122.18 (Gima 22067) - S5122.20 (Gima 22068) - S5122.22 (Gima 22069) - S5122.24 (Gima 22070) - S5123.18 (Gima 22077) - S5123.20 (Gima 22078) - S5123.22 (Gima 22079) - S5123.24 (Gima 22080)

Jiangsu Suyun Medical Materials Co., Ltd. No.18  
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang  
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA  
Made in China

Shanghai International Holding Corp. GmbH (Europe)  
Eiffestraße, 80, 20537, Hamburg, Germany

**Gima S.p.A.**  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

 **0123**

 **STERILE EO**

 **MD**

**Product name:**

Sterile Urethral Catheter for Single Use (Foley catheter)

Type: 1-way,2-way,3-way

**Sizes:**

Straight Tip/ Standard tip-

**Intended purpose:**

For temporary emiction by inserting it to bladder through urethra when patients can not micturate by themselves.

**Intended users:**

It can be used by trained doctors, nursing staff, patients or accompanying personnel.

**Structural Composition:**

The product is composed of drainage funnel, inflation access port, valve, irrigation access port and tube.

**Indications:**

Urinary obstruction and urine retention,ect.

**Contraindications:**

Those who are allergic to latex materials should not use this product.

**Patient group:**

Patients with urinary retention caused by urinary tract obstruction.

Rescue critically ill patients.

Patients in need of diagnosis and treatment of bladder disease.

People undergoing surgery.

Patients with spinal cord injury.

**Performance Characteristics:**

a. The balloon shall not leak and shall not occlude the lateral drainage holes.

b. The drainage tunnel shall not part from the test connector.

c. The tip/shaft union and lateral drainage holes shall not show any sign of breaking and neither the tip nor the funnel shall become detached from the shaft..

**Method:**

a. Wash your hands and put on gloves to clean around the urethral opening. Those who have foreskin should pull the foreskin down to expose the urethral opening and clean it.

b. After preparing for catheterization, open the individual package from the opening.

c. Lubricate the catheter with medical liquid paraffin or medical silicone oil.

d. Insert the urinary catheter into the bladder slowly through the urethra for normal catheterization.

Any residual risk and any undesirable side-effects:none

Installation information:none

**Clinical benefits:**

The product help patients to temporary emiction.

**Caution:**

a. When using the product, it should strictly implement the requirements of aseptic operation specifications and relevant regulations, and can be used by trained doctors, nursing staff, patients or accompanying personnel. However, when patients or accompanying staff conduct urinary catheterization for the first time, please use it under the guidance of professional medical staff. In the course of use, the operator or patient should be treated in a timely manner according to medical regulations.

b. If you find any possible quality doubt during use, please stop using it immediately and notify our company quickly.

c. The product should be sterilized with EO, and the product should be sterile.

d. The product is disposable and should be destroyed after use.

e. Please checking the individual package before using it.

f. Don't use if package is opened or damaged.

g. Sterile and validity for 5 year. Don't use if overdue.

h. This product needs to be used in conjunction with a container for urine. When using it, it should avoid splashing urine and polluting the environment.

i. Discard after use. After use, the products shall be collected, transported, stored and disposed to ensure that the products are harmless to human health and the environment. Or the product can be disposed in accordance with the local medical regulations for wastedisposal.

j. If the product is used in a household environment, after use, the product shall be disposed as household hazardous waste.

k. All serious incidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located.

**Storage:**

The product should be stored in a well-ventilated room temperature.

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos - índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Symboli-indeksi - Indeks simbolov - Index simbolov - Index de simbol - Symbol index - Indeks simbola - Szimbólum index - Symbolindex - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sümbole indeksi

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