Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

PROFESSIONAL MEDICAL PRODUCTS

LAME GOUGE IN CARBONIO
CARBON GOUGE BLADES
LAMES GOUGE EN ACIER AU CARBONE
CUCHILLAS DE GUBIA DE CARBONO
SKALPELL-KLINGEN AUS KARBON
LÁMINAS DE GOIVA DE CARBONO
SKALPELE ZE STALI WĘGLOWEJ
SZÉNACÉL PEDIKÚRPENGÉK
HIILITALTATERÁT
UGLJENIČNA SJEČIVA
SKALPELLBLADER I KARBONSTÁL
KOOLSTOFSTAAL GUTSMESJES
LAME DALTA DIN OTEL CARBON
KOLFIBERBLAD
ЛЕЗВИЯ ДЛЯ ДОЛОТА ИЗ УГЛЕРОДИСТОЙ СТАЛИ
ОСТРИЕТА ОТ ВЪГЛЕРОДНА СТОМАНА ЗА ИЗДЪЛБАВАНЕ
ΛΑΜΕΣ ΑΠΟ ΑΝΘΡΑΚΑ
شفرات مجوب من الكربون

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso
Gebrauchs- und instandhaltungsanleitung - Guia para utilização
Instrukcja obsługi - Használati utasítás - Käyttöohje - Korisnički Priručnik
Brukerhåndbok - Gebruikershandleiding - Manual De Utilizare
Användarhandbok - Руководство Пользователя - Ръководство За
Потребителя - Οδηγίες χρήσης - **شيفر /تجهيزات**

ENGLISH

PRODUCT

Surgical Blade (Sterile)
(Carbon Steel and Stainless Steel)
Sterilized by Gamma radiation of minimum 25 kGy and not more than 32 kGy
(2.5 M Rads to 3.2 M.Rads)

MATERIALS USED

Stainless Steel, Carbon Steel

INDICATIONS

- Dissection Incision.
- Different size for different thickness of tissue & sight.
- Removal of extra tissue from operating area.

CONTRAINDICATIONS

- Re-used of blades can work as carrier for communicable disease to patient and/or user.
- Adverse event may be happen if blades are used after expiry date as expiry date of product is expiry date of sterility.

INSTRUCTIONS FOR USE

- Select proper type and size of Blade as per the sight, always use bigger size for thick tissue and smaller for fine soft tissue.
- After selection clean the sight with proper disinfectant
- Check the package of blade for its intactness, check expiry date of the blade and then remove blade from package.
- Blade has to attached on corresponding pre sterile B P handle as per the size slot and check that handle is well fixed in the slot of blade.
- with grip on metal handle choose the sight for incision.
- Incision has to place as per the desired length preferably measured and marked.

WARNINGS

- Read instruction for use
- The product should be used only by a qualified surgeon, Doctor or paramedic.
- Before use always check integrity of product and packing along with expiry date.
- For single use only, **If re-used this can work as carrier for communicable disease, HIV, Hepatitis, contagious dieses, undue diseases to patient and/or user.**
- Use product immediately after opening the pack.
- GIMA is not responsible for any possible consequences resulting from improper use.
- GIMA do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is not guaranteed if packet is broken/torn.
- Re-sterilization and re-use of blade cause to change in mechanical properties and material used.
- Re-sterilization and re-use of blade may not meet the intended use as blade may be blunt.

STORAGE CONDITION

- Keep away from direct sun light.
- Keep away from rain
- Storage temperature should be 10 to 40 degree celcius
- Humidity of storage area should be 35% RH to 65% RH.
- Keep away from children.
- Store in cool and dry place.

DISPOSAL OF USED POUCH AND BLADES

- After use, pieces and pouches shall be disposed off in compliance with the local regulatory requirement of handling medical waste.

PRECAUTIONS

- Always open the pouch from peel apart direction to avoid injury.
- Devices are extremely sharp, use care while handling
- Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by Gamma radiation will become unsterile. Proper procedures must be used as applicable for handling any sterile product.
- Care must be taken during disposal of device to avoid any contact or injury due to the sharp nature of the device.
- Proper care must be taken while choosing the right blade for the right handle to avoid any injury or accident.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application

DAMAGED PRODUCT RETURN

- Damaged product should be return immediately to the source from where it is purchased.

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REF 22501 - 22502 - 22503 - 22504 - 22505
22606 - 22508 - 22510 - 22512 - 22515

Importatore / Importer:
Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) - Italy

Paramount Surgimed Limited
A-106, RIICO Industrial Area,
Bhiwadi-301019, Alwar, Rajasthan, India

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MDSS GmbH