CURETTE DERMATOLOGICA MONOUSO CON RASCHIETTO
DISPOSABLE DERMAL CURETTE WITH SCRAPER
CURETTE DERMATOLOGIQUE AVEC GRATTOIR JETABLE
CURETTE DERMICA DESECHABLE CON RASPADOR
EINWEG-HAUTKÜRETTE MIT SCHABER
CURETA DERMICA DESCARTÁVEL COM RASPADOR
JEDNORAZOWY ŁYZECZKA SKÓRNA ZE SKROBACZKĄ
EGYSZER HASZNÁLATOS, DERMÁLIS KÜRET KAPARÓVAL
KERTAKÄYTTÖINEN IHOKYVETTI, JOSSA KAIVIN
JEDNOKRATNA DERMALNA KIRETA SA STRUGAČEM
HUDCURETTE MED SKRAPE TIL ENGANGSBRUK
DERMATOLOGISCHE CURETTE VOOR EÉNMALIG GEBRUIK MET
SCHRAPER
CHIURETĂ DERMICĂ DE UNICĂ FOLOSINȚĂ CU VÂRF PENTRU RĂZUIRE
ENGÅNGS DERMAL CURETTE MED SKRAPA
ОДНОРАЗОВАЯ КОЖНАЯ КЮРЕТКА СО СКРЕБКОМ
ДЕРМАЛНА КЮРЕТА СЪС СКРЕПЕР ЗА ЕДНОКРАТНА УПОТРЕБА
ΚΙΟΥΡΕΤΑ ΔΕΡΜΑΤΟΣ ΜΙΑΣ ΧΡΗΣΗΣ ΜΕ ΞΕΣΤΡΟ
قطش كم دم ح اووالا لامل عتسائل دي دلج فتسكم

Manuale d'uso - User manual - Manuel de l'utilisateur - Guia de uso
Gebrauchs- und Instandhaltungsanleitung - Guia para utilização
Instrukcja obsługi - Használati utasítás - Käyttöohje - Korisnički Priručnik
Brukerhåndbok - Gebruikershandleiding - Manual De Utilizare - Använd-
grihandbok - Руководство Пользователя - Ръководство За Потребителя
- Οδηγίες χρήσης -

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ENGLISH

PRODUCT

Curette Dermal
(Size: 2mm, 3mm, 4mm, 5mm, 7mm).
Sterilized by Gamma radiation of minimum 25 kGy and not more than 32 kGy
(2.5 M Rads to 3.2 M.Rads).
MATERIALS USED
Stainless Steel, ABS Plastic / LDPE.

INDICATIONS

- Different size for different thickness of tissue & location.
- Removal of skin from operating area for test specimen.

CONTRAINDICATIONS

- Re-use of blades can work as carrier for communicable disease to patient and/or user.
- Adverse event may be happen if blades are used after expiry date as expiry date of product is expiry date of sterility.

INSTRUCTIONS FOR USE

- Select proper type and size of Dermal Curette as per the location, always use bigger size for thick tissue and smaller for fine soft tissue.
- After selection clean the location with proper disinfectant.
- Inspect package of Dermal Curette for its intactness expiring and then remove blade from package.
- With grip on handle choose the location for taking skin specimen.

WARNINGS

- Read instruction for use.
- The product should be used only by a qualified surgeon, Doctor or paramedic.
- Before use always check integrity of product and packing along with expiry date.
- For single use only, if re-used this can work as carrier for communicable disease, HIV, Hepatitis, contagious diseases, undue contamination which can create some diseases to patient and/or user.
- Use product immediately after opening the pack.
- Gima is not responsible for any possible consequences resulting from improper use.
- Gima do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is not guaranteed if packet is broken/torn.
- Re-sterilization and re-use of blade cause to change in mechanical properties and material used.
- Re-sterilization and re-use of blade may not meet the intended use as blade may be blunt.

STORAGE CONDITION

- Keep away from direct sun light.
- Keep away from rain.
- Storage temperature should be 10 to 40 degree celcius.
- Humidity of storage area should be 35% RH to 65% RH.
- Keep away from children.
- Store in cool and dry place.

DISPOSAL OF USED POUCH AND BLADES

- After use, pieces and pouches shall be disposed off in compliance with the local regulatory requirement of handling medical waste.

PRECAUTIONS

- Always open the pouch from peel apart direction to avoid injury.
- Devices are extremely sharp, use care while handling.
- Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by Gamma radiation will become unsterile. Proper procedures must be used as applicable for handling any sterile product.
- Care must be taken during disposal of device to avoid any contact or injury due to the sharp nature of the device.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

DAMAGED PRODUCT RETURN

- Damaged product should be return immediately to the source from where it is purchased.
- In case of adverse event please contact EC representative

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REF 22562 - 22563 - 22564
22565 - 22567

Importatore / Importer:
Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) - Italy

Paramount Surgimed Limited
A-106, RIICO Industrial Area,
Bhiwadi-301019, Alwar, Rajasthan, India

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It's Interventional Regulatory limited
Albreda House, Lydgate Lane
South Yorkshire, Sheffield
S10 5FH, England, United Kingdom

2460

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MDSS GmbH