Gima S.p.A.  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)

PROFESSIONAL MEDICAL PRODUCTS

#### **CURETTE PER BIOPSIA MONOUSO**

**DISPOSABLE BIOPSY PUNCH**

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**PUNZON PARA BIOPSIA DESECHABLE**

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#### **PRODUCT**

Biopsy Punch

(Size: 1mm, 1.5mm, 2mm, 2.5mm, 3mm, 3.5mm, 4mm, 5mm, 6mm, 7mm,8mm,10mm, 12mm).

Sterilized by Gamma radiation of minimum 25 kGy and not more than 32 kGy (2.5 M Rads to 3.2 M.Rads).

#### **MATERIALS USED**

Stainless Steel, ABS Plastic / LDPE.

#### **INDICATIONS**

- For obtaining diagnostic full-thickness skin specimen.
- Different size for different skin area of body.
- Removal of skin specimen from operating area.

#### **CONTRAINDICATIONS**

- Re-used of blades can work as carrier for communicable disease to patient and/or user.
- Adverse event may be happen if blades are used after expiry date as expiry date of product is expiry date of sterility.

#### **INSTRUCTIONS FOR USE**

- Select proper type and size of Biopsy Punch as per skin area and size of specimen.
- After selection clean the location with proper disinfectant.
- Inspect package of Biopsy Punch for its intactness expiring and then remove blade from package.
- With grip on handle choose the skin for cutting skin specimen.
- Punching has to be done by rotating punch on skin with the gentle pressure.
- Remove the Biopsy Punch after cutting the desire depth of skin.
- Take the sample of specimen.

#### **WARNINGS**

- Read instruction for use.
- The product should be used only by a qualified surgeon, Doctor or paramedic.
- Before use always check integrity of product and packing along with expiry date.
- For single use only. **If re-used this can work as carrier for communicable disease, HIV, Hepatitis, contagious diseases, undue contamination which can create some diseases to patient and/or user.**
- Use product immediately after opening the pack.
- GIMA is not responsible for any possible consequences resulting from improper use.
- GIMA do not hold any responsibility if device re-used or re-sterile.
- Sterility of product is not guaranteed if packet is broken/torn.
- Re-sterilization and re-use of blade cause to change in mechanical properties and material used.
- Re-sterilization and re-use of blade may not meet the intended use as blade may be blunt.

#### **STORAGE CONDITION**

- Keep away from direct sun light.
- Keep away from rain.
- Storage temperature should be 10 to 40 degree celcius.
- Humidity of storage area should be 35% RH to 65% RH.
- Keep away from children.
- Store in cool and dry place.

#### **DISPOSAL OF USED POUCH AND BIOPSY PUNCH**

- After use, Biopsy punch and pouches shall be disposed off in compliance with the local regulatory requirement of handling medical waste.

#### **PRECAUTIONS**

- Always open the pouch from peel apart direction to avoid injury.
- Devices are extremely sharp, use care while handling.
- Care must be taken so that the pouch is not opened in an unsterile area otherwise the blade which has already been sterilized by Gamma radiation will become unsterile. Proper procedures must be used as applicable for handling any sterile product.
- Care must be taken during disposal of device to avoid any contact or injury due to the sharp nature of the device.
- In case of changes in the performance of device for intended use replace the defective device by new to full fill the required application.

#### **DAMAGED PRODUCT RETURN**

- Damaged product should be return immediately to the source from where it is purchased.

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**Paramount Surgimed Limited**  
A-106, RIICO Industrial Area,  
Bhiwadi-301019, Alwar, Rajasthan, India

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Gima S.p.A. Via Marconi, 1 - 20060 Gessate (MI) - Italy
**MDSS GmbH**