CURETTE STIEFEL





(EN) Read instructions carefully before use.





(EN) Available sizes 4 & 7 mm





(EN) Sterile. For SINGLE USE only.

If multiple biopsies or samples are needed on the

- same patient, a new device must be used each time.

 → Do not re-sterilise or re-use due to risk of infection.
 - Do not use if the sterile blister package is compromised or damaged.

(EN) Curette is a device for tissue removal, including excision of superficial lesions.

2.

a.

(EN) Two techniques are recommended for use of the curette. Select the appropriate technique for the




(EN) Technique 1, which is more commonly used. Stretch skin with fingers of non-dominant hand. Hold curette between thumb, index and middle finger with the sharp side at the edge of the lesion.



a.ii.

(EN) Technique 2, which due to reduced precision may be more suitable for hardened or tough skin. Hold curette in distal inter-digital fold of index finger, supported by other fingers of the curetting hand with the sharp side at the edge of the lesion. Place thumbs to provide a stable base.

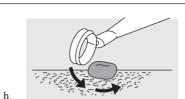

(EN) Gently guide curette, so that piece of tissue can be removed in one incision.

3.



(EN) Sterilised with Ethylene Oxide by: sfm medical devices GmbH, Brückenstrasse 5, D-63607 Wächtersbach.



GSK Consumer Healthcare (GMDT) Clocherane, Youghal Road, Dungarvan, Co. Waterford, Ireland



(EN) See pack or carton for: **LOT**







(GB) GlaxoSmithKline UK Ltd, 980 Great West Road, Brentford, Middlesex, TW8 9GS, UK.

05/2020 05_SF201_03_21