

BIOPSY PUNCH




(EN) Read instructions carefully before use.

(EN) Sterile. For SINGLE USE only.
If multiple biopsies or samples are needed on the same patient, a new device must be used each time.
→ Do not re-sterilise or re-use due to risk of infection.
→ Do not use if the sterile blister package is compromised or damaged.

(EN) Available sizes:
2; 3; 3.5; 4; 5; 6 & 8 mm


1.

(EN) Biopsy punch is a standardised method for performing skin biopsies.

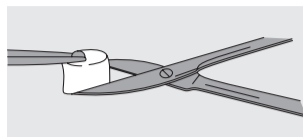

c.


2.

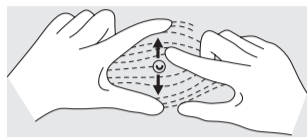

a.

(EN) Stretch skin, to relax skin tension lines.

(EN) Gently grip and raise skin sample with small anatomical tweezers, avoiding any crushing. Separate with scissors or scalpel at base if necessary.

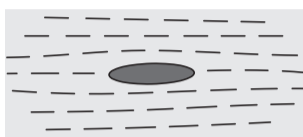

d.

(EN) For wounds ≤ 4 mm gently compress or close with hydrocolloid tape. For wounds ≥ 4 mm close with sutures, clips or adhesive strips as clinically appropriate.



b.

(EN) Hold punch between thumb and index finger. Place on upper skin surface, with moderate pressure press down in a clockwise "screwing" motion.

3.



(EN) Sterilised with Ethylene Oxide by:
sfm medical devices GmbH, Brückenstrasse 5, D-63607 Wächtersbach.



GSK Consumer Healthcare (GMDT)
Clocherane, Youghal Road,
Dungarvan,
Co. Waterford,
Ireland



4.

(EN) See pack or carton for:



5.

(GB) GlaxoSmithKline UK Ltd, 980 Great West Road,
Brentford, Middlesex, TW8 9GS, UK.