



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

## **PAD ESERCITAZIONE SUTURA - SUTURE TRAINING PAD COUSSIN D'ENTRAÎNEMENT AUX SUTURES ALMOHADILLA DE ENTRENAMIENTO DE SUTURA ALMOFADA PARA TREINO DE SUTURA**

**Material:** Silicone

**Materiale:** Silicone

**Matériau:** Silicone

**Material:** Silicona

**Material:** Silicone

**Size:** 17.2x12x1.2cm

**Dimensioni:** 17,2x12x1,2 cm

**Dimensions:** 17,2x12x1,2 cm

**Dimensiones:** 17,2x12x1,2 cm

**Dimensões:** 17,2x12x1,2 cm

**Weight:** 240g

**Peso:** 240 g

**Poids:** 240 g

**Peso:** 240 g

**Peso:** 240 g

### **Suturing techniques**

The suture pad provide several types of wounds. You can practice following skills:


**1. Simple interrupted suture:** simple operation, the most application, each stitch a single knot, used in the skin, subcutaneous tissue, muscle, aponeurosis suture, especially for wounds with infection.




**2. Continuous suture method:** after the first needle is sutured, the entire wound is sutured with the suture, and a needle before the end is left, and the weight tail is pulled out to remain on the opposite side to form a double line and a heavy line tail knot.




**3. Continuous seam stitching method:** the operation saves time, the haemostatic effect is good, the thread is interlaced every time during the suture process, and is mostly used for closing the gastrointestinal end and suturing during skin transplantation.

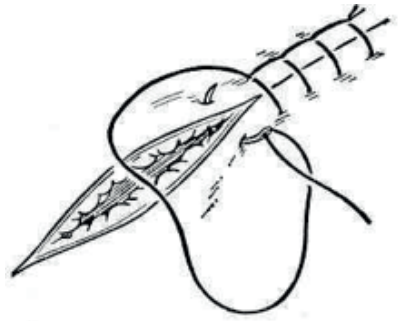

**4. 8 - word suture:** consists of two intermittent sutures, which are firm and time-saving, such as sutures of the fascia.

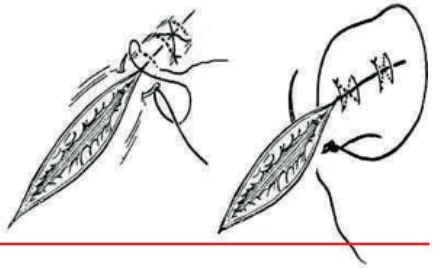

**5. Subcuticular suture:** the needle is inserted from one end of the incision, and then alternately passes through the inner edge of the incision edge of both sides, and is sewn to the other end of the incision to be pierced, and finally tightened. Commonly used for suturing of exposed skin incisions, such as cervical thyroidectomy.

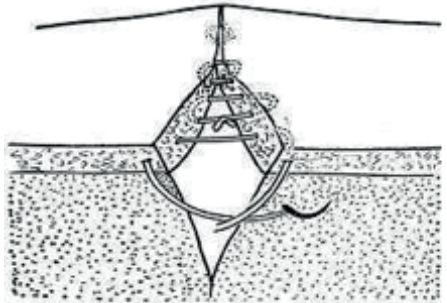

**CONDIZIONI DI GARANZIA GIMA**

Si applica la garanzia B2B standard Gima di 12 mesi.

**GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

**CONDITIONS DE GARANTIE GIMA**

La garantie appliquée est la B2B standard Gima de 12 mois.

Fabbricante/Manufacturer/Fabricant/Fabricante/Fabricante  
Hangzhou Haizhu MIM products Co., Ltd.  
No.868 shixin north Road, Hangzhou, 311215  
Made in China

**REF B3361**

**CONDICIONES DE GARANTÍA GIMA**

Se aplica la garantía B2B estándar de Gima de 12 meses.

**CONDIÇÕES DE GARANTIA GIMA**

Aplica-se a garantia B2B padrão GIMA de 12 meses.

Importato da / Imported by / Importé de  
Importado de / importado por:  
**Gima S.p.A.** Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
**www.gimaitaly.com** - Made in China



**Gima 23020**