meso-relle ®

Raccordo multiniettore

Needles-syringe connector multinjector Raccord seringue-aiguilles multipuncteur Nadel -Spritzen- Verbindung für Mehrfachinjektionen Conector multiinyector para aguja-jeringa

Instructions for use

The Mesorelle connector enables more than one needle to be used with a single syringe aflowing a multiple injection to be made at any one time.

The Mesorelle connector is mounted on the syringe after the latter has been filled.

Next the needles are inserted and any residua1 air is expelled in the same manner as with a single-needle syringe. Models available are linear 3, 5 or circular a can accomodate 3, 5, 7 Luer fitting

The material used is special trans- parent plastic desi-gned for medical purposes and allowing the inner chamber to be checked. Sterility garanteed if package intact. Products must be stored in a cool and dry place, avoid direct sun light.

M.V. srl – Via Don G. Dossetti 5/7 – 46023 Palidano di GONZAGA (MN) Distributor: BIOTEKNE srl, - Via della Bastia, 9 – 40033 Casalecchio di Reno (BO)





Ultimo aggiornamento: Maggio 2011