

INSTRUCTIONS FOR USE

HYPODERMIC NEEDLES WITH SAFETY MECHANISM

PURPOSE OF USE:

Sterile, non-toxic, non-pyrogenic, for single use, needle for injections and administration of medicinal solutions.

INSTRUCTIONS FOR USE:

Before using check the expire date.
Check the integrity of the packaging before use
Open the packaging and use immediately.
Do not use if packaging is damaged.
In case of re-use, infections can occur: consult a doctor.
Prepare the area carefully and aseptically.
Remove the needle from the blister.
Go on with the therapy.
Activate the safety mechanism.
Dispose of it properly after use.

LATEX FREE:

The device is totally latex free.

STERILIZATION METHOD:

Sterilized by Ethylene Oxide (EtO), residual amount of gas in accordance with the regulations.

DISPOSAL:

Disposal must be made in accordance with the applicable government regulations. Incineration or landfill disposal must be carried out under controlled conditions. The product that did not come in to contact with infectious agents or other hazardous materials, cannot be considered dangerous waste.

CONSERVATION:

Store in a cool dry place and far away from the any source of heat at a temperature.

*These Instructions for Use note must be preserved by the user for
the duration of the MD.*



Health&Safety

RAYS SPA

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - www.rays.it - info@rays.it