

INSTRUCTIONS FOR USE

INJ-LIGHT SYRINGES WITH HYPODERMIC NEEDLE

PURPOSE OF USE

Sterile, non-toxic, non-pyrogenic, for single use, syringe with mounted hypodermic needle for injections and administration of solutions.

INSTRUCTIONS FOR USE:

Before using check the expire date. Do not use if packaging is damaged. In case of re-use, infections can occur: consult a doctor.

Prepare the area carefully and aseptically. Remove the syringe from the blister.

Go on with the administration.

Dispose of it properly after use.

LATEX FREE:

The device is latex free to prevent any risk of allergic reaction.

STERILIZATION METHOD:

Sterilized by Ethylene Oxide (EtO), residual amount of gas in accordance with the regulations.

DISPOSAL:

Disposal must be made in accordance with the applicable government regulations. Incineration or landfill disposal must be carried out under controlled conditions. The product that did not come in to contact with infectious agents or other hazardous materials, cannot be considered dangerous waste.

CONSERVATION:

Store in a cool dry place and far away from the any source of heat at a temperature.

These Instructions for Use note must be preserved by the user for the duration of the DM.

Ed. 3 – May 2023



Health&Safety

RAYS SPA

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - www.rays.it - info@rays.it