

# ***INSTRUCTIONS FOR USE***

## ***INJ-SAFE SYRINGES WITH MOUNTED SAFETY HYPODERMIC NEEDLE***

### **PURPOSE OF USE**

Sterile, non-toxic, non-pyrogenic, for single use, syringe with mounted safety hypodermic needle for injections and administration of solutions.

### **INSTRUCTIONS FOR USE:**

Before using check the expire date. Do not use if packaging is damaged. In case of re-use, infections can occur: consult a doctor.

Prepare the area carefully and aseptically. Remove the syringe from the blister. Pull down the protection device, pushing it on the side, until a “click” is heard, assuring the opening lock. Go on with the administration. After finishing, put up the protection device until the needle has been completely covered and a “click” is heard. It means that the device has been completely covered in an irreversible way. It is also possible to finish the connection process on a table, in order to avoid any contact with the needle. Dispose of it properly after use.

### **LATEX FREE:**

The device is latex free in order to prevent any risk of allergic reaction.

### **STERILIZATION METHOD:**

Sterilized by Ethylene Oxide (EtO), residual amount of gas in accordance with the regulations.

### **DISPOSAL:**

Disposal must be made in accordance with the applicable government regulations. Incineration or landfill disposal must be carried out under controlled conditions. The product that didn't come in to contact with infectious agents or other hazardous materials, can not be considered dangerous waste.

### **CONSERVATION:**

Store in a cool dry place and far away from the any source of heat at a temperature.

*These Instructions for Use note must be preserved by the user for the duration of the DM.*

*Ed. 3 – May 2023*



**Health&Safety**

**RAYS SPA**

via Francesco Crispi - 60027 Osimo (AN)

Tel. 071 2868935 - Fax 071 2868911 - [www.rays.it](http://www.rays.it) - [info@rays.it](mailto:info@rays.it)