
***DISPOSITIVO ASPIRA VELENO
POISON EXTRACTION TOOL
POMPE À VENIN
DISPOSITIVO PARA ASPIRACIÓN DE VENENO***

HOW TO USE Unstinger™

As soon as possible after the bite, place your Unstinger™ over the affected zone and apply a gentle pressure in order to assure optimal adherence to your skin. Use only the red adapter tip depending on the type of bite or the interested area.

Start extracting by slowly pulling back the winglets of the red piston and keeping a constant vacuum: the substance which was injected into your body through the bite (venom, saliva, etc.) will be rapidly extracted. After the piston has reached its endpoint, the action may be repeated several times.

WARNINGS AND PRECAUTIONS

- Unstinger™ is not a toy and its small components may be easily ingested by younger children: we therefore recommend not to leave your Unstinger™ unattended if and when newborns or young children are around.
- The use of Unstinger™ never excludes the need to consult a physician in the event of allergy or particular sensitivity to the stings of certain insects/animals.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

ENGLISH

Unstinger™ is a second-generation, hand-operated vacuum pump to be used after bites and stings by common insects (bees, mosquitos, hornets).

REF Unstinger™ (Gima 24395)

Distribuito da / Distributed by / Distribué par / Distribuido por:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Made in Denmark
