

## User Manual of FiT129D Mini Breath Alcohol Checker

Thanks for purchasing our pocket size Alcohol Checker, FiT129D and it gives a digital reading of your breath alcohol content and thus your possible concentration of intoxication.

### Instruction

- 1) Install 2 x 1.5V AAA Alkaline batteries into the battery compartment
- 2) Wash your mouth with WATER before using. Abstain from smoking or drinking alcohol for at least 15 minutes.
- 3) Pressing the START Key 0.5 second to turn on the power, sensor pre-heating will be activated for an approximate 14 seconds and the display counts down.
- 4) "BLOW" flashes with a beep tone when count down to zero.
- 5) Press & hold the START Key and blow continuously for approximately 6 seconds. "BLOW" and keep the tester 1/2 inch (12mm) away from the mouth while blowing. stop blowing when you hear a single beep tone and the reading will then be shown on the display.
- 6) When low battery indicator flashes, the reading is not valid, please replace the batteries immediately.

Many factors (eg. environment gas residue, smoke & various smell in human mouth, etc...) may affect the BAC level, so the readings are For Reference Only.

Neither the reseller nor manufacturer accept any liability arising from the use of the unit

Don't drink before you drive!

