




GIMA

PROFESSIONAL MEDICAL PRODUCTS

GONIOMETRO PER LE DITA 6" STD FINGER GONIOMETER 6" STD GONIOMÈTRE POUR DOIGTS 6" STD GONIÓMETRO DE DEDOS 6" STD

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso

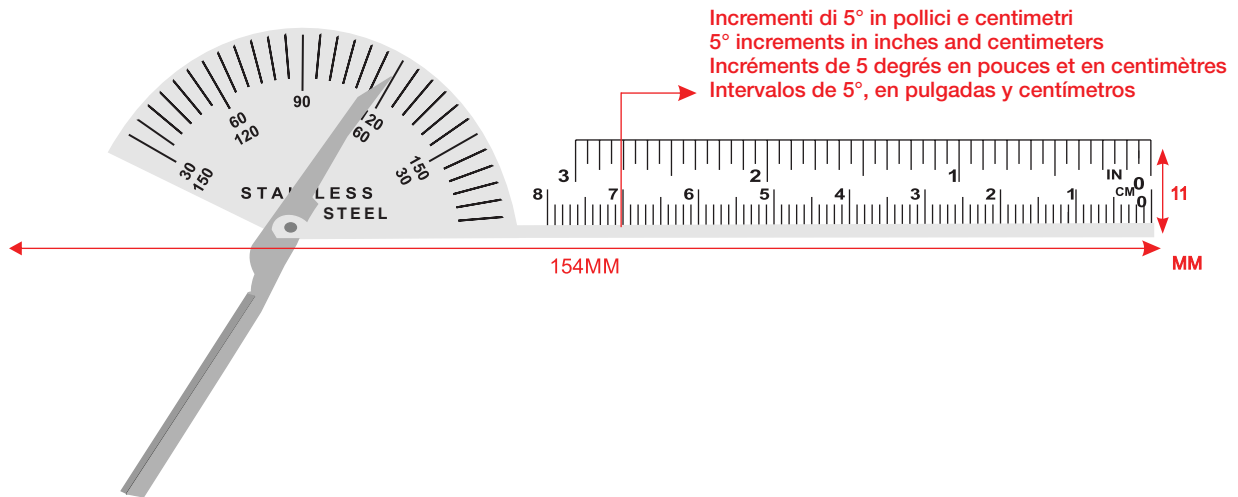

Tipo di acciaio - Steel Type
Type d'acier - Tipo de acero:
AISI 430

Le misure si intendono indicative
The measures are intended as indicative
Les mesures sont destinées à être indicatives
Las medidas tienen carácter indicativo

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.
ATTENTION: The operators must carefully read and completely understand the present manual before using the product.
AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.
ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

GIMA 24900

Fabbricante, Manufacturer, Fabricante, Fabricante:
JIMCO INDUSTRIES
4-KM Aimen Abad Road Near Akbar Abad Chowk Bhadal,
Sialkot-51310-Punjab - Pakistan - info@jimcoind.com
Made in Pakistan

REF RE-160-10

Importato da / Imported by / Importé par/ Importado por:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



GB - INSTRUCTIONS OF USE

A finger goniometer measures range-of-motion (ROM) of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes. Standard 6" stainless steel finger goniometer has head with opposing 180° to 30° scales in 5° increments. Linear scale on arm measures in inches and centimeters.

INDICATION OF USE & PRECAUTIONS

Those devices are designed for professional use by qualified operators with necessary skills.





The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion.

Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content.

This product is intended to provide easy-to-read indicative measurements.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

Simboli / Symbols / Symboles / Simbolos					
	IT - Leggere e seguire attentamente le istruzioni per l'uso GB - Read instructions carefully FR - Lire et suivre attentivement la notice ES - Leer y seguir atentamente las instrucciones de uso		IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar	LOT	IT - Numero di lotto (vedi scatola / bustina) GB - Lot number (see box / package) FR - Numéro de lot (voir boîte/sachet) ES - Número de lote (ver caja / sobre)
	IT - Leggere attentamente le istruzioni per l'uso GB - Please read instructions carefully FR - Lire et suivre attentivement la notice ES - Leer atentamente las instrucciones de uso		IT - Conservare in luogo fresco e asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco	REF	IT - Codice prodotto GB - Product code ES - Código producto FR - Code produit