



PROFESSIONAL MEDICAL PRODUCTS

GONIOMETRO DIGIT 1 CON DESIGN PER LE DITA DIGIT 1 FINGER DESIGN GONIOMETER GONIOMÈTRE NUMÉRIQUE POUR DOIGT DIGIT 1 GONIÓMETRO DE DEDOS DIGIT 1

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso

Le misure si intendono indicative
The measures are intended as indicative
Les mesures sont destinées à être indicatives
Las medidas tienen carácter indicativo

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.
ATTENTION: The operators must carefully read and completely understand the present manual before using the product.
AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.
ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

GIMA 24901


Fabbricante, Manufacturer, Fabricante, Fabricante:
JIMCO INDUSTRIES
4-KM Aimen Abad Road Near Akbar Abad Chowk Bhadal,
Sialkot-51310-Punjab - Pakistan - info@jimcoind.com
Made in Pakistan

REF RE-220-10

Importato da / Imported by / Importé par/ Importado por:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



Incrementi di 5° in pollici e centimetri
 5° increments in inches and centimeters
 Incréments de 5 degrés en pouces et en centimètres
 Intervalos de 5°, en pulgadas y centímetros

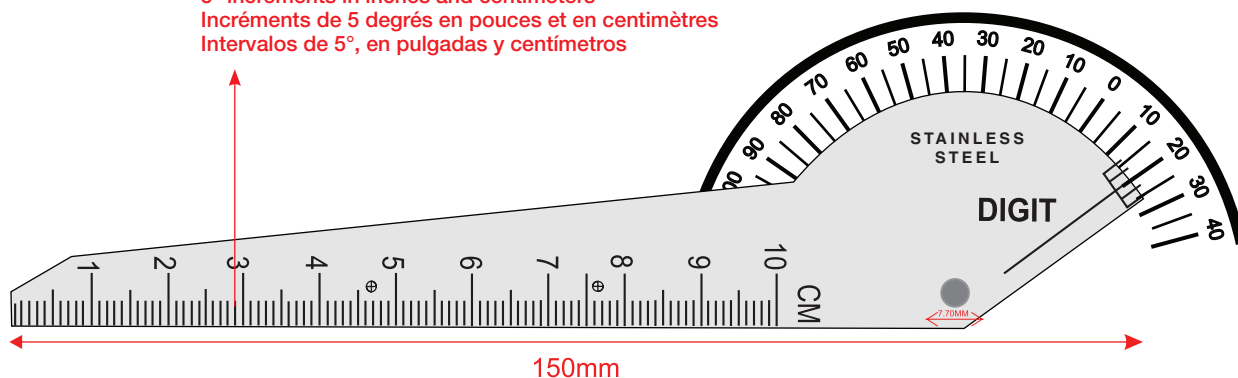

Tipo di acciaio - Steel Type - Type d'acier - Tipo de acero: **AISI 430**

GB - INSTRUCTIONS OF USE

A finger goniometer measures range-of-motion (ROM) of finger joints (metacarpophalangeal and interphalangeal) and other small joints including toes. Unique design allows measurements to be taken by the therapist using only one hand; this frees up the therapist's other hand to stabilize the patient during measurement or to record the measurement; the standard finger goniometer requires two hands to make a finger ROM measurement. 6" Stainless steel finger goniometer has head with opposing measures 110° flexion through 40° hyper-extension in 5° increments. Linear scale on arm measures in inches and centimeters and has 10cm / 4" scale.

INDICATION OF USE & PRECAUTIONS

Those devices are designed for professional use by qualified operators with necessary skills.

The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion.

Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content.

This product is intended to provide easy-to-read indicative measurements.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies