




GIMA

PROFESSIONAL MEDICAL PRODUCTS

GONIOMETRO 14" BRACCIA EXTRA LUNGHE 180° 180° EXTRA LONG ARMS GONIOMETER 14" GONIOMÈTRE 14" 180° BRAS EXTRA LONGS GONIÓMETRO DE BRAZOS EXTRA LARGOS 180° - 14"

Manuale d'uso - User manual - Manuel de l'utilisateur - Guía de uso

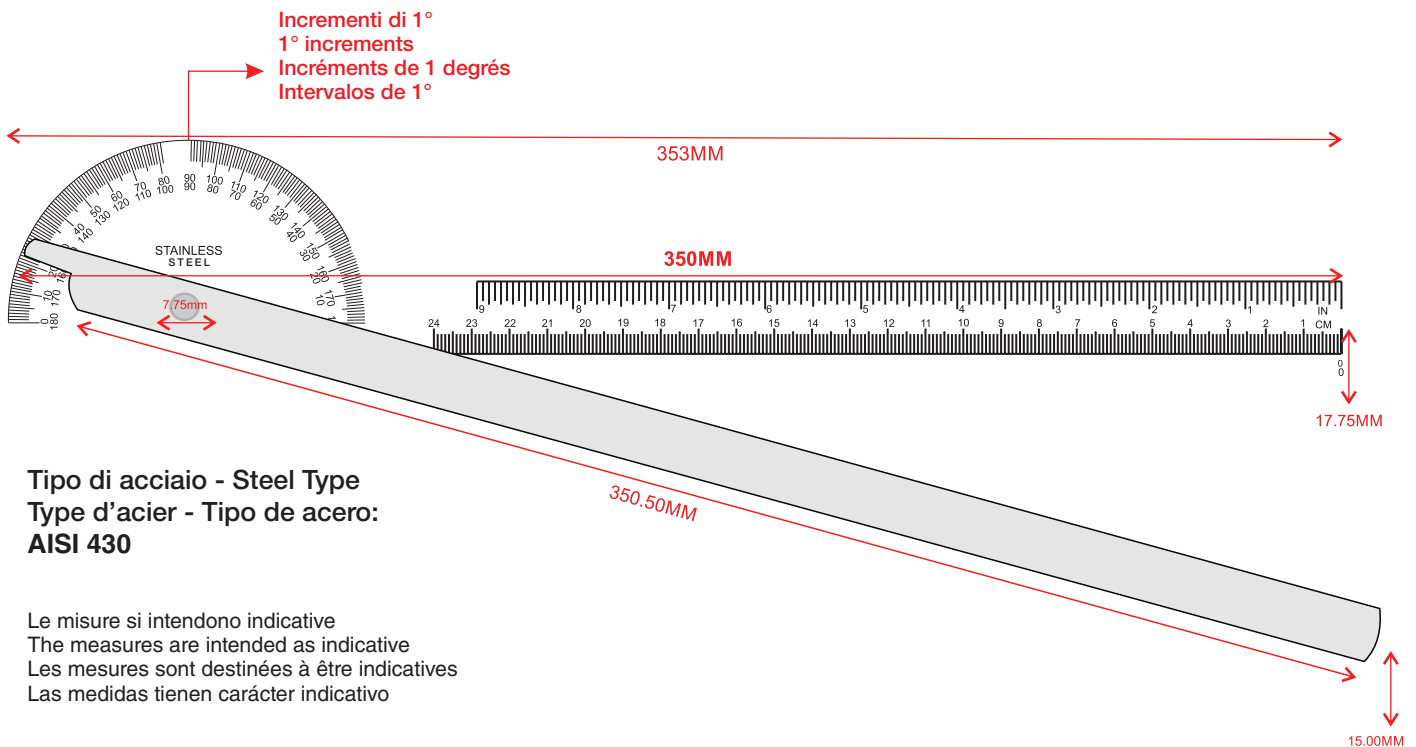

Tipo di acciaio - Steel Type
Type d'acier - Tipo de acero:
AISI 430

Le misure si intendono indicative
The measures are intended as indicative
Les mesures sont destinées à être indicatives
Las medidas tienen carácter indicativo

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.
ATTENTION: The operators must carefully read and completely understand the present manual before using the product.
AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.
ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

GIMA 24904

Fabbricante, Manufacturer, Fabricante, Fabricante:
JIMCO INDUSTRIES
4-KM Aimen Abad Road Near Akbar Abad Chowk Bhadal,
Sialkot-51310-Punjab - Pakistan - info@jimcoind.com
Made in Pakistan

REF RE-200-10

Importato da / Imported by / Importé par/ Importado por:
Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



GB - INSTRUCTIONS OF USE

A goniometer measures range-of-motion (ROM) of various joints. The 180° head is marked in 1° increments. Tension between the arms is controlled by the thumb knob.

Two opposing scales-extra-long arms for measuring all joints.

INDICATION OF USE & PRECAUTIONS

Those devices are designed for professional use by qualified operators with necessary skills.





The use of the product involves direct contact with patient's skin; always wear protective gloves, do not use on dull skin, handle carefully to avoid accidental abrasion. Clean and sterilize before each use; use neutral non-aggressive disinfectant solution (PH7), do not use solutions containing chloride ions or high alcohol content.

This product is intended to provide easy-to-read indicative measurements.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

Simboli / Symbols / Symboles / Símbolos

	<p>IT - Attenzione: Leggere e seguire attentamente le istruzioni (avvertenze) per l'uso GB - Caution: read instructions (warnings) carefully FR - Attention: lisez attentivement les instructions (avertissements) ES - Precaución: lea las instrucciones (advertencias) cuidadosamente</p>		<p>IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil ES - Conservar al amparo de la luz solar</p>	<div style="border: 1px solid black; padding: 2px; display: inline-block;">LOT</div>	<p>IT - Numero di lotto GB - Lot number FR - Numéro de lot ES - Número de lote</p>
	<p>IT - Leggere le istruzioni per l'uso GB - Consult instructions for use FR - Consulter les instructions d'utilisation ES - Consultar las instrucciones de uso</p>		<p>IT - Conservare in luogo fresco e asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec ES - Conservar en un lugar fresco y seco</p>	<div style="border: 1px solid black; padding: 2px; display: inline-block;">REF</div>	<p>IT - Codice prodotto GB - Product code ES - Código producto FR - Code produit</p>