

OCCHIALI X5-PRO -ANTIAPPANNANTI E ANTIGRAFFIO
X5-PRO GOGGLES - FOG RESISTANT AND ANTI-SCRATCH
LUNETTES X5-PRO - ANTI-BUÉE ET ANTI-RAYURES
GAFAS X5-PRO: RESISTENTES AL VAHO Y ANTIARAÑAZOS
ÓCULOS X5-PRO - RESISTENTES AO NEVOEIRO E ANTI-RISCOS
X5-PRO-BRILLE – BESCHLAGFREI UND KRATZFEST
X5-PRO-BRIL - CONDENSBESTENDIG EN KRASBESTENDIG
X5-PRO BRILLE - TÅGEAFVISENDE OG ANTI-RIDSE
X5-PRO GOGGLES - IMMA RESISTENTA OCH ANTI-REPOR
X5-PRO SUOJALASIT - HUURTUMATON JA NAARMUUNTUMATON
X5-PRO BRILLE - TÅKEBESTANDIG OG ANTI-RIPE
GOGLE X5-PRO - ODPORNE NA MGŁĘ I ZARYSOWANIA
BRÝLE X5-PRO - ODOLNÉ PROTI ZAMLŽENÍ A PROTI POŠKRÁBÁNÍ
X5-PRO SZEMÉVVEZETÉK - PÁRA- ÉS KARCÁLLÓ
GOGGLES X5-PRO - REZISTENTI LA CEATA SI ANTI-ZGARIETURI
NAOČALE X5-PRO - ODPORNE NA MAGLJENJE I OGREBOTINE
OČALA X5-PRO - ODPORNA NA ROSENJE IN PRASKE
ОЧИЛА X5-PRO - УСТОЙЧИВИ НА МЪГЛА И ПРОТИВ НАДРАСКВАНЕ
X5-PRO GOGGLES - ANΘΕΚΤΙΚΑ ΣΤΗΝ ΟΜΙΧΛΗ ΚΑΙ ΑΝΤΙΧΑΡΑΚΤΙΚΗ
شده ل ل ة داضم و بابض ل ل ة م واقم - X5-PRO تاراظن

REF GDG01 (Gima 25257 - 25258)

Zhejiang Gongdong Medical Technology Co., Ltd
No. 10 Beiyuan Ave., Huangyan Taizhou,
318020 Zhejiang China
www.chinagongdong.com
Made in China



ZOUSTECH S.L.
Pso. Castellana, 141- Planta 19, 28046-Madrid,
Spain

Importato da / Imported by / Importé par / Importiert von/ Importado por/ Importado por/ Εισάγεται
από / Importowane przez / Importat de / Importováno / Importerad av / Maahantuoja / Importerad av /
Uvezao / Importálja / Importert av / Geïmporteerd door / Importeret af / Внесен от / داري تاسا :

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



Instructions for Use

Model: Ordinary Type

Rev. A/1 Effective:2022-10-17

Description

The Medical isolation goggles is intended to create a physical barrier separating the eyes from the immediate surroundings. This device prevents liquids or droplet splash from contacting the user. The Medical isolation goggles is intended for single use.

Indication for Use:

Medical isolation goggles are personal protective equipment intended for protection of the associated mucous membranes eyes from splashes, sprays, and spatter of body fluids.

Contraindications:

None.

Product side effects:

This product has no side effects.

Warning:

1. Do not use if the components show any sign of damage.
2. Do not expose components to temperature greater than listed on package labeling.
3. Dispose of the product as biohazardous waste per institutional protocol.












Operation:

Take out the goggles from the packaging bag, confirm that the goggles in good condition, adjust the telescopic leg of the goggles to a comfortable position and wear it on the ear.

Storage:

Store product in a dry place at 0 - 30°C.
Keep away from sunlight.

Explanations for symbols

	Expiration date		Disposable device, do not re-use
	Lot number		Don't use if package is damaged
	Date of manufacture		Keep away from sunlight
	Medical Device compliant with Regulation (EU) 2016/425		Keep in a cool, dry place
	Manufacturer		Authorized representative in the European Community
	Product code		

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies

CE PPE Cat.II issued by:

DIN CERTCO Gesellschaft für Konformitätsbewertung mbH Notified Body
0196, Alboinstraße 56, 12103 Berlin, Germany
according to the Annex II of Regulation (EU)2016/425 - Harmonized EN
166: 2001 Directive 2001/95/EC