



GIMA

Gima S.p.A.

Via Marconi, 1 – 20060 Gessate (MI) Italy

gima@gimaitaly.com – export@gimaitaly.com

www.gimaitaly.com

PROFESSIONAL MEDICAL PRODUCTS

DIGITAL VETERINARY THERMOMETER

Operating Instruction:

Directions:

1. Turn on the thermometer by pressing the button. The thermometer will beep, after
2. sound is stop, meaning the unit is now ready to use.
3. Gently insert the thermometer tip into rectum or into the food.
4. Without 6 seconds the thermometer will beep and display the temperature reading.
5. Shut off automatically after 15 seconds without any operation.

Technical Specifications:

- Instant temperature in 6 seconds.
- Beeps indicating.
- LCD display & Memory.
- Display temperature in both Fahrenheit or Celsius scale.
- Easily operation.
- Reliable & sanitation & safe.
- Shut off automatically after 15 seconds.
- Suit for measuring kinds of animal, like dog, cat, pig, cattle and so on.

Measurement Range:

(32.00-44.00)°C, (89.6-111.2)°F

Accuracy

+/- 0.1°C between 35.00°C to 39.00°C

+/- 0.2°F between 95°F to 102.20°F

+/- 0.2°C below 35.00°C to over 39.00°C

+/- 0.4°F below 95.00°F to over 102.20°F

Battery type: 2x1.5V (AAA)DC

Battery life: approx. 5000 measurement readouts Dimensions: 190x110x35mm

REF 25588 / KDV-03



Kellyunion Electronics Co., Ltd
No. 114, Five Star Village, Jiangkou Town
Hanjiang District 3511115 Putian, Fujian
PEOPLE'S REPUBLIC OF CHINA
Made in China

EC **REP**

Landlink GmbH
Dorfstrasse 2/4, Emmendingen,
79312 Germany

