











**GB Instructions for use** ( **i**): Body temperature indicator with encapsulated liquid crystals ( **MD**). First use: remove the protective transparent film from the front of the indicator. Place the thermometer for 15 seconds on a dry forehead, holding it by both ends. If the letter N is visible there is no fever. If a number appears, the temperature corresponds to the visible value. If two numbers are visible, the temperature is a value in between. If temperature rises, consult a doctor. Use the indicator at room temperature, with adequate brightness, under adult supervision for children under 13. Clean the indicator by rubbing it gently with soapy water for at least 30 seconds or with alcohol-free skin disinfectants (do not soak). External use on healthy skin. Do not expose to sunbeams (, keep away from heat sources, alcohol and solvents. (Issue of June 17, 2020).

Explanation of symbols:

CE0051 CE mark and identification of notified body.  Identifies the name of the manufacturer (may be followed by the manufacturing date).  Identifies the manufacturing date.  Identifies the name of the distributor. **LOT** Product lot number. **REF** Device model.  Consult the instructions for use.  Do not expose to sunlight or UV sources. **MD** Medical Device.  Do not dispose of in the environment. Dispose of consciously.  Polycoupled material. Mixed waste collection. °C Degrees centigrade. S1 Support symbol



Made by: I.P.S. S.r.l. - Via Civesio, 6 - 20097 San Donato Mil.se (Mi) - Italy +39 025279641

[www.ips-srl.it](http://www.ips-srl.it)