

48h Fevermonitor


48 Ore di controllo continuo dello stato febbrile

48 Hours continual monitoring of the body's temperature


Veloce 30 secondi
Fast 30 seconds


Sicuro
Safe

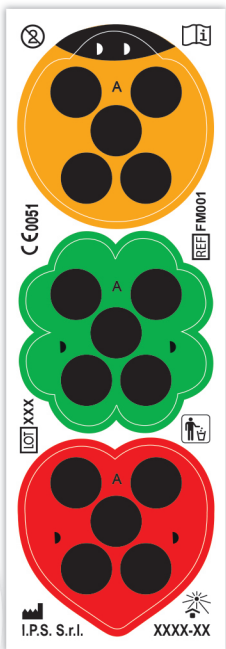


Dispositivo medico
Medical device





ips
The innovative world of microencapsulated products

MADE IN ITALY



DIRECTIONS FOR USE: Liquid crystal body temperature indicator. Remove the indicator from its support and make it adhere well to the dry forehead. After 30 seconds, if numbers appear after the letter "N" (no fever), the temperature corresponds to the highest one. If a fever is read, a clinical thermometer should be used. Carry out the test at room temperature, with adequate brightness, under adult supervision for children under 13. Do not expose to direct sunlight () , heat, alcohol and solvents. The device is disposable () . External use on healthy skin. Do not keep in continuous contact with the skin for more than 48 hours. (Issue of May 22, 2019).

ISTRUZIONI PER L'USO: Indicatore di temperatura corporea a cristalli liquidi. Staccare l'indicatore dal supporto e farlo aderire bene alla fronte asciutta. Trascorsi 30 secondi, se oltre alla lettera "N" (assenza di febbre), appaiono più numeri, la temperatura corrisponde al valore più alto. In caso di febbre, è consigliato l'uso di un termometro clinico. Eseguire il test a temperatura ambiente, con luminosità adeguata, sotto la supervisione di un adulto per bambini sotto i 13 anni. Non esporre ai raggi del sole () , e fonti elevate di calore, alcool e solventi. Il dispositivo è monouso () . Uso esterno su cute sana, non tenere in contatto continuo con la cute per più di 48 ore. (Emissione del 22 maggio 2019).

Legenda simboli:  Modello di dispositivo;  N. di lotto del prodotto;  Marchio CE;  Consultare istruzioni for use;  Product for number;  CE mark;  Non di spazzare nell'ambiente. Raccolta differenziata - Plastica;  Manufacturer and production Date;  Manufacturer and production Date;

LOT 122



 Made by: I.P.S. S.r.l. - Via Civesio, 6
20097 San Donato Milanese (MI) - Italy +39 025279641
www.ips-srl.it - 2020-04

REF FM001 C€0051