

**SCHERMO FACIALE ANTI-NEBBIA MONOUSO - ANTI-FOG DISPOSABLE FACE SHIELD - ÉCRAN FACIAL ANTI-BUÉE JETABLE - PANTALLA FACIAL DESECHABLE ANTINEBLA - PROTETOR FACIAL DESCARTÁVEL ANTI-FOG - ANTI-BESCHLAG-EINWEG-GESICHTSSCHILD - ΑΝΤΙΟΜΙΧΛΙΚΗ ΑΣΠΙΔΑ ΠΡΟΣΩΠΟΥ ΜΙΑΣ ΧΡΗΣΗΣ - JEDNORAZOWA OSŁONA TWARZY ANTYFOGOWA - ECRAN FACIAL DE UNICĂ FOLOSINȚĂ ANTI-ABURIRE - JEDNORÁZOVÝ OBLIČEJOVÝ ŠTÍT PROTI ZAMLŽOVÁNÍ - ANSIKTSVISIR MED ANTI-IMBILDNINGSFUNKTION FR ENGÅNGSBRUK - KERTÄKÄYTTÖINEN HUURTEENESTOKA-SVOSUOJA - JEDNORAZOVÝ TVÁROVÝ ŠTÍT ODOLNÝ VOČI ZAHMLIEVANIU - ANTICONDENS GELAATSSCHERM VOOR EENMALIG GEBRUIK - JEDNOKRATNI VIZIR PROTIV MAGLJENJA - PÁRAMENTES EGYSZER HASZNÁLATOS ARCVÉDŐ - PÁRAMENTES EGYSZER HASZNÁLATOS ARCVÉDŐ - DUGFRI ANSIGTSSKÆRM TIL ENGANGSBRUG - ЛИЦЕВ ЦИТ ПРОТИВ ЗАМЪГЛЯВАНЕ ЗА ЕДНОКРАТНА УПОТРЕБА - NERASOJANTIS VIENKARTINIS APSAUGINIS VEIDO SKYDELIS - VIENREIZĒJĀS LIETOŠANAS PRETAIZSVĪŠANAS SEJAS AIZSARGS - UDUVASTANE ŪHEKORDNE NĀOKAITSE - TÄKEHEMMENDE ANSIKTSKJERM TIL ENGANGSBRUK - درم لامل عتس الل بابض لل داضم هجو ورد**

**ENGLISH Description:** The anti-fog face shield. is composed of a piece of PET film with a piece of protection film, a foam support and an elastic band.

**Indication for Use:** The anti-fog face shield is used to help protect the who e face against potential contamination from blood pathogens, body fluids, or harmful chemical splash during inspection or treatment.

**Contraindications:** None.

**Product side effects:** This product has no side effects.

**Warning:**

1. Exposing m high humid envirom11ent may cause losing efficacy of anti-fog effect
2. Suspension of use in case of damage or slackness during use.
3. During use, please try not to touch the center of the lens.
4. Materials that may come into contact with the wea1er1s skin could cause allergic reactions in sensitive individuals.

**Operation:**

Step 1: Remove the inner protective film.

Step 2: Pull the elastic band and put on the head

**Storage:** store product in a dry p1ace at 0 - 30°C.

Keep away from sunlight

**GIMA WARRANTY TERMS** The Gima 12-month standard B2B warranty applies

