



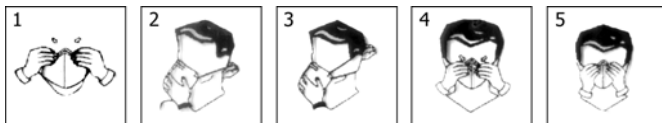

Prodotto da: **Eleco srl**
Produced by: **Via Verdi 11**
Produit par: **24121 Bergamo – ITALY**

I Semi-maschere filtranti monouso
GB Single-use filtering half-masks
D Partikelfiltrierenden Einweg-Halbmasken
F Demi-masques filtrants jetables
E Semimáscaras filtrantes desechables
P Semimáscaras filtrantes de utilização única
TR Tek kullanımlık filtreli yarım yüz maskeleri
NL Halvegeleatsmasker met filter voor eenmalig gebruik
S Filtrerande halvmasker för engångsbruk
FIN Suodattavat kertakäyttöiset puolinaamarit
CZ Filtrační polomaska na jedno použití
PL Maski ochronne jednorazowego użytku
GR Ημιμάσκες διήθησης μιας χρήσης
HR Polu-maske s filtrom za jednokratno upotrebu

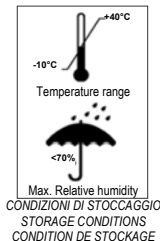

ENGLISH INSTRUCTIONS FOR USE

IMPORTANT: Single use filtering half-masks here described are classified into three classes - FFP1, FFP2, and FFP3. Prior to use, check to which class the half-mask belong to, according to EN149:2001+A12:2009 standard and check the appropriateness for the field of use - the indication regarding the class to which the product belong to is reported on the packaging and on the half-mask. The deadline for a safe use of the item is reported on the package.

APPLICATIONS: protection against particles, as well as against solid and liquid aerosol (e.g. oil-mists)



Be careful to check the completeness of the mask prior to use it. If you notice laceration and/or stain, ask for a replacement.
NR symbol reported on the marking indicates that the half-mask is not reusable and must be discarded at the end of the shift or session.
D symbol indicates that the mask passed the clogging test performed with the use of particles of dolomite.
The protection provided by an FFP3 or FFP2 filtering half-mask includes that provided by FFP1 class.

FITTING INSTRUCTIONS:

1. Place a finger inside the half-mask. Bend the nose clip around the finger to confer to the shape the profile of the nose.
2. While holding the half-mask in your hand, put on the inferior branch of the elastic-band over the head behind the nape, and adjust the elastic-band underneath the ears.
3. Take the upper branch of the elastic-band and place it over the head, being careful to adjust the elastic-band over the ears.
4. Using both hands, shape the nose clip (if on place) around nose and adjust the tension of the upper and lower portions of the elastic-strap by means of the appropriate buckle when on place, or simply moving the strap in a comfortable position so as to ensure a good adhesion to the face.
5. (a) To test fit for masks without exhalation valve: Cup both hands over the mask as on picture 5 and exhale vigorously. (b) To test fit for masks provided of exhalation valve: Cup both hands over the mask as on picture 5 and inhale sharply. If air flows from mask edge to the outside or if it comes in from the edges, you should re-position the mask over the face and if appropriate tighten the strap when provided of buckles.
6. Change the mask immediately if breathing becomes difficult or if the mask becomes damaged, or if there are problems of adhesion to the face. Careful observance of these instructions is essential for a safe use of the respirator.

WARNING: [PPE – Personal Protective Equipment]

If it is not possible to obtain an adequate adhesion of the mask to the face, do not enter on places of potential risk. / Failure to observe the prescriptions reported on to this leaflet may lead to an inappropriate use of this PPE and consequently cause sickness or death. / A properly selected mask is essential to protect your health. Before using this PPE, consult an Industrial Hygienist or Occupational Safety Professional to determine the suitability for your intended use. / This product does not supply oxygen. Use only in adequately ventilated areas containing sufficient oxygen to support life. Limits regarding oxygen content on air for a given field of use of these masks may vary from one country to another. We suggest to take as reference the parameters defined in the nation of use. Nevertheless, Eleco s.r.l. advise against the use of these PPEs in areas where oxygen concentration is less than 19.5%. / Do not use when concentrations of contaminants are immediately dangerous to health or life. / Do not use in explosive atmospheres. / Leave work area immediately and return to fresh air if (a) breathing becomes difficult, or (b) dizziness or other distress occurs. / Facial hairs or beards and certain respiratory conditions and characteristics may reduce the effectiveness of this filtering half-masks. / Never alter or modify this filtering half-mask in any way. / Only for single use. No maintenance necessary. Discard the used half-mask after single use. / Keep the half-masks which have not been used in their original package away from direct sunlight until use. – The model identification code is reported on the package of the product.