

PILL PULVERIZER




Easily pulverize large or small pills

- 1) Place pill in cavity of clean section
- 2) Screw down blue top section all the way, that's all there is to it !
- 3) Dissolve powder in milk, juice ...



CAUTION: do not crush time-release medication, if in doubt, consult your pharmacist or physician.

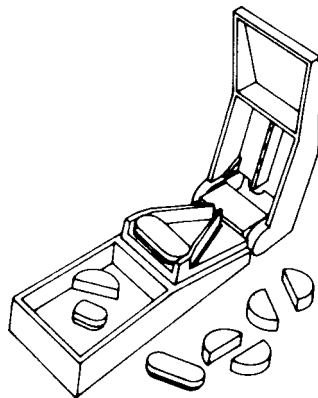

PILL SPLITTER



Easily divides large or small pills

- 1) Set on firm surface
- 2) Place pill inside "V" so pill touches both sides of "V"
- 3) Lower the lid and press lightly
- 4) Store unused portions in base



CAUTION: sharp blade, keep away from children



GIMA

TAGLIAPILLOLE - FRANTUMAPILLOLE
PILL SPLITTER - PILL PULVERIZER
COUPE-PILULE - BROYEUR DE PILULES
CORTAPÍLDORAS - TRITURADOR DE PÍLDORAS
CORTA-COMPRIMIDOS - QUEBRA-COMPRIMIDOS
ΚΟΜΜΑΤΙΑΣΤΗΣ ΧΑΠΙΩΝ - ΚΑΤΑΣΤΡΟΦΕΑΣ ΧΑΠΙΩΝ

مكسر لأقراص الدواء - قاطع لأقراص الدواء

ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.

ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

ATENÇÃO: Os operadores devem ler e entender completamente este manual antes de usar o produto.

ΠΡΟΣΟΧΗ: Οι χειριστές αυτού του προϊόντος πρέπει να διαβάσουν και να καταλάβουν πλήρως τις οδηγίες του εγχειριδίου πριν από την χρήση του.

الحذر: على العمال قراءة وفهم هذا الدليل بكامله قبل البدء باستعمال المنتج.



M25730-M-Rev.2-03.10

GIMA Spa - Via Marconi, 1 - 20060 Gessate (MI) - Italia

ITALIA: Tel. 199 400 401 (8 linee r.a.) - Fax 199 400 403

E-mail: gima@gimaitaly.com - www.gimaitaly.com

INTERNATIONAL: Tel. ++39 02 953854209 - Fax ++39 02 95380056

E-mail: export@gimaitaly.com - www.gimaitaly.com