



GIMA

PROFESSIONAL MEDICAL PRODUCTS

**FRANTUMAPILLOLE
PILL PULVERIZER
BROYEUR DE PILULES
TRITURADOR DE PÍLDORAS
TABLETTENZERKLEINERER
ESMAGADOR DE PÍLULAS
KRUSZARKA DO TABLETEK
ΚΑΤΑΟΤΡΟΦΕΑΣ ΧΑΠΙΩΝ**

GIMA 25731

Fabbricante/Manufacturer/Fabricant/Fabricante
Fabricante/Hersteller/ProducentΚατασκευαστής
Ningbo Pinbo Plastic Manufactory Co., LTD
6F, No.17 Building, 636 Duantang East Rd, Ningbo, 315012 China
Made in China

REF **PT97644**

M25731-M-Rev.4-11.24


Importato da / Imported by / Importé par
Importado por / Importado por / Importiert von
Importowane przez / Εισαγωγή από:

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com



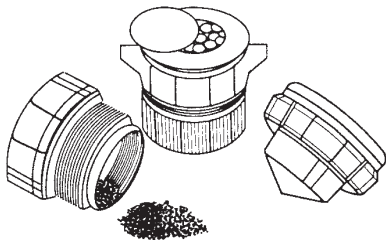

ENGLISH

PILL PULVERIZER

Easily pulverize large or small pills.

- 1) Place pill in cavity of clean section.
- 2) Screw down blue top section all the way, that's all there is to it!
- 3) Dissolve powder in milk, juice ...

CAUTION: do not crush time-release medication, if in doubt, consult your pharmacist or physician.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.