

TORNIQUETS

super-tex

Tubolar model

200 pieces



Medical device - Class I Italian

Law Decree n°46 24/02/97 Implementation of EEC Directive 93/42

ISTRUZIONI FOR USE

Typical uses:

Supertex latex rubber tourniquets may be used to compress blood vessels during blood sampling or as to stop haemorrhaging in the case of injury.

Insitruction:

Apply tourniquet on position and tight. Before to remove reliaise gently.

Restrictions:

Don't use more then 20 minutes. Don't use for patients whit latex sensitivity. Use only for arm and thigh.

Conservation:

Supertex tourniquets should be kept according to normal procedures.

Disposal:

Supertex tourniquets may be burned or disposed in waste disposal units whitout any danger to the enviroment.

Expiration: 5 years from the production date if correctly conserved.

Super-Tex srl

Via S. Giuliano, snc 03039 Sora (Fr) Tel. 0776.890198 Fax 0776.892110

email: super-tex@libero.it