PROCTOSCOPIO STERILE MONOUSO
STERILE PROCTOSCOPE FOR SINGLE USE
RECTOSCOPE STÉRILE À USAGE UNIQUE
PROCTOSCOPIO ESTÉRIL DESECHABLE
PROCTOSCÓPIO ESTÉRIL PARA UTILIZAÇÃO ÚNICA
STERILES PROKTOSKOP FÜR DEN EINMALIGEN GEBRAUCH
ΑΠΟΣΤΕΙΡΩΜΕΝΟ ΠΡΩΚΤΟΣΚΟΠΙΟ ΜΙΑΣ ΧΡΗΣΗΣ
STERYLNY REKTOSKOP JEDNORAZOWEGO UŻYTKU
STERILNÍ PROKTOSKOP NA JEDNORÁZOVÉ POUŽITÍ
STERILT PROKTOSKOP FÖR ENGÅNGSBRUK
STERIILI KERTAKÄYTTÖINEN PROKTOSKOOPPI
STERILNI PROKTOSKOP ZA ENKRATNO UPORABO
STERILNÝ PROKTOSKOP NA JEDNO POUŽITIE
PROCTOSCOPIUM STERIL DE UNICĂ FOLOSINȚĂ
STERIELE PROTOSCOOP VOOR EENMALIG GEBRUIK
STERILNI PROKTOSKOP ZA JEDNOKRATNU UPOTREBU
EGYSZER HASZNÁLATOS, STERIL PROKTOSZKÓP
STERILT PROKTOSKOP TIL ENGANGSBRUG
СТЕРИЛЕН ПРОКТОСКОП ЗА ЕДНОКРАТНА УПОТРЕБА
STERILUS VIENKARTINIS PROKTOSKOPAS
STERILS PROKTOSKOPS VIENREIZĒJAI LIETOŠANAI
STERIILNE PROKTOSKOOP ÜHEKORDSEKS KASUTAMISEKS

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REF 7231 02 (Gima 25957)

Jiangsu Suyun Medical Materials Co., Ltd. No.18
Jin Qiao Road Dapu Industrial Park 222002 Lianyungang
JIANGSU PROVINCE, PEOPLE'S REPUBLIC OF CHINA
Made in China

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße, 80, 20537, Hamburg, Germany

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

CE 0123

STERILE EO

MD

Product name:

Sterile Proctoscope for single use
Type: I, II, III, IV

Size:

Small(14#), Medium(22#), Large(26#)

Intended purpose:

Sterile Proctoscope for single use is mainly applied to inspection and anal lesion surgery.

Intended users:

Sterile Proctoscope for single use is used by trained doctors, nursing staff.

Structural composition:

Sterile Proctoscope for single use is composed of outer tube , inner tube, light source connector.

Indications:

The product only plays a common tool for anorectal disease examination and treatment.

Contraindications: None

Patient group: patients with anorectal disease.

Performance characteristics:

single-use and sterile product

Method of use:

Open the primary package, The inner tube into the outer tube, together into the rectum, drawing out the inner tube, connecting the light source, and then observe the patient's condition.

Any residual risk and any undesirable side-effects: none

Installation information: none

Clinical benefits:

Sterile Proctoscope for single use help medical staff to check and operate on patients with anorectal diseases

 **Caution:**

- A. The product is for single use only. If the product re-use, can occur cross-infection.
- B. The product is sterile, sterilized by Ethylene Oxide Gas, and valid for five years. After expiration, it is prohibited to use.
- C. When using , insert the product slowly into the rectum, please stop using if the patient feels pain.
- D. Do not use if the small package is broken or unintentionally opened before use or contains foreign matter.
- E. The use of the product must comply with the requirements of the relevant operating standards and relevant laws and regulations of the medical department, and is used by trained doctors or nursing personnel. In case of any physical abnormality of the operator or user during the use process, it should be dealt with in time according to the medical standards.
- F. If you find any quality problem during use, please stop using it immediately and inform us immediately.
- G. Discard after use. After use, the products shall be collected, transported, stored and disposed to ensure that the products are harmless to human health and the environment. Or the product can be disposed in accordance with the local medical regulations for waste disposal.

Storage:

The product should be stored in a room with no corrosive gas and well ventilated, with adequate protection for the product

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Indice dei simboli - Symbol index - Index des symboles - Índice de símbolos- índice de símbolo - Symbolindex - Ευρετήριο συμβόλων - Indeks symboli - Index symbolů - Symbol index - Simboli-indeksi - Indeks simbolov - Index de simbol - Symbool index - Indeks simbola - Szimbólum index - Symbolindeks - Индекс на символа - Simbolių rodyklė - Simbolu rādītājs - Sümboleite indeks

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