



Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

PROFESSIONAL MEDICAL PRODUCTS

BACINELLE IN PLASTICA AUTOCLAVABILI
AUTOCLAVABLE PLASTIC BASINS
CUVES EN PLASTIQUE STÉRILISABLES EN AUTOCLAVE
CUBETAS DE PLÁSTICO PARA AUTOCLAVE
BACIAS EM PLÁSTICO AUTOCLAVÁVEIS
KUNSTSTOFFSCHALEN IN AUTOKLAVEN STERILISIERBAR
ΠΛΑΣΤΙΚΑ ΔΟΧΕΙΑ ΠΟΥ ΑΠΟΣΤΕΙΡΩΝΟΝΤΑΙ ΣΕ ΑΥΤΟΚΑΥΣΤΟ
أحواض بلاستيكية قابلة للتعقيم بالأوتوكلاف

Manuale d'uso - User manual - Manuel de l'utilisateur - Guia de uso - Guia para utilização
Gebrauchs- und instandhaltungsanleitung - Οδηγίες χρήσης - دليل الإستعمال والرعاية

ENGLISH

Before using the product, sterilise it.

The product is autoclavable up to a maximum temperature of 137°C, but before performing this operation it is advisable to wash it in a dishwasher according to ISO 15883 standards. Place the clean and free product in such a way that there is nothing preventing the correct circulation of water and steam both during washing and sterilisation. Avoid dry sterilisation and the use of solvents or abrasive cleaning substances; only use products that are compatible with polypropylene. Do not store below 4°C. Product life about 36 months.



At the end of the product life cycle, comply with current regulations regarding disposal. Do not disperse into the environment!

REF

26630 - 26631 - 26632 - 26633 - 26635
26636 - 26638 - 26639 - 26640



Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) Italy
Made in UK

