

WASHING AND CLEANING INSTRUCTIONS of
MULTICUFF BAG - 3 pediatric cuffs - 27177
GIMA MULTICUFF BAG - 3 cuffs – 27178
MULTICUFF BAG - 5 cuffs - 27179

MATERIAL

The products are mainly composed of polyester 600D.

CLEANING

1. For cleaning product surface from minor stains, you may use alcohol swab or cloth moistened with alcohol to wipe its surface.
2. For disinfecting product surface, you may use alcohol swab by applying it to product surface you wish to disinfect.
3. Do not use bleach or solvents.

WASHING

1. To preserve product's visual, the recommended washing method is by hand.
2. If you were to use washing machine, set the washing cycle to the gentlest one. Make sure that hook and loop fastener are fastened first and remove the shoulder strap.
3. Use cold or room temperature water with mild detergent. Avoid bleach.
4. Gently scrub product surface especially on parts with printing.
5. Rinse thoroughly with water until all detergent residue is removed.
6. Hang the product to dry. Avoid direct sunlight.