



GIMA

PROFESSIONAL MEDICAL PRODUCTS

BILANCIA DIGITALE IN VETRO
GLASS DIGITAL SCALE
PÈSE-PERSONNES EN VERRE À AFFICHAGE NUMÉRIQUE
GLASPERSONENWAAGE
BÁSCULA DIGITAL DE CRISTAL
BALANÇA DIGITAL DE VIDRO
ΨΗΦΙΑΚΗ ΖΥΓΑΡΙΑ ΑΠΟ ΓΥΑΛΙ
SZKLANA WAGA CYFROWA
چاگز ن م یلای جید نازی م
DIGITAALINEN LASIVAAKA
STAKLENA DIGITALNA VAGA
ÜVEGBŐL KÉSZÜLT DIGITÁLIS MÉRLEG
CÂNTAR DIGITAL DIN STICLĂ
DIGITAL GLASVÁG

MANUALE D'USO E MANUTENZIONE
USE AND MAINTENANCE BOOK
INSTRUCTIONS DE FONCTIONNEMENT ET ENTRETIEN
BETRIEBS UND WARTUNGS ANWEISUNGEN
MANUAL DE USO Y MANTENIMIENTO
MANUAL DE USO E MANUTENÇÃO
ΕΓΧΕΙΡΙΔΙΟ ΧΡΗΣΗΣ ΚΑΙ ΣΥΝΤΗΡΗΣΗΣ
PODRĘCZNIK EKSPLOATACJI I KONSERWACJI
KÄYTTÖ- JA HUOLTO-OPAS
PRIRUČNIK ZA UPOTREBU I ODRŽAVANJE
HASZNÁLATI ÉS KARBANTARTÁSI ÚTMUTATÓ
MANUAL DE UTILIZARE ȘI ÎNTREȚINERE
ANVÄNDNINGSG- OCH UNDERHÅLLSMANUAL

REF GBS-810D (GIMA 27248 - 27249)

Fabbricato da/Made by/Faite par/Hecho por/Feito por/Hergestellt von/Zrobione przez/Ἐτιασμένο από/
GUANGDONG TRANSTEK MEDICAL ELECTRONICS.CO.LTD.
Jin'an Road , Minzhong, Zhongshan 528441, China
Made in China

Importato e distribuito da/Imported and distributed by
GIMA S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com



FEATURES & SPECIFICATION


- Auto ON
- Auto OFF
- Weight Units:kg,lb,st:lb
- Capacity;150kg/330lb/23st:8lb
- Graduation: 0.1kg/0.2lb/0.2lb(1lb,≥20st)
- Overload indication: “ — — — — ”
- Low battery indication “Lo”
- Power supply: 1 x CR2032 battery

LCD Full Display




TO CHANGE BATTERY

- Ensure +/- terminals are correct way around.
 - Use proper tool to help release the battery from the compartment if necessary.
- Please remove the insulating strip before use.

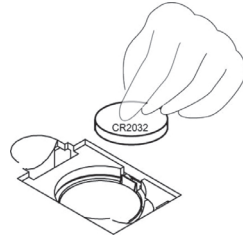

FUNCTION KEYS

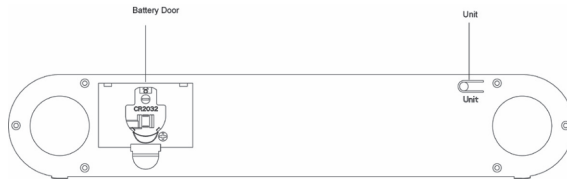

OPERATION INSTRUCTIONS

1. Press UNIT key to switch on and select unit.
2. Place the scale on the solid and flat ground ,let the scale turn off automatically.
3. User stand on the platform of scale directly, it will flicker and lock on the LCD after the weight steady, then automatically turn off after 10 seconds.
4. If without any operation , it'll turn off automatically
5. When you don't use it ,please balanced lay it, then do not put any item on the scale.



KINDLY REMINDER:

Don't use scale on the carpet

In order to ensure weighing accurate, when you move the scale, please first turn on (or tap on) and wait for turn off, then, you can stand on the platform again and measure your weight. Otherwise,it will add scale own weight.



DISPOSAL:

The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment. For further information on recycling points contact the local authorities, the local recycling center or the shop where the product was purchased. If the equipment is not disposed of correctly, fines or penalties may be applied in accordance with the national legislation and regulations.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.