



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**BILANCIA VETERINARIA - MEDIA**  
**DIGITAL VET SCALE - MEDIUM**  
**BALANCE VÉTÉRINAIRE DIGITALE - MOYENNE**  
**BÁSCULA VETERINARIA - MEDIA**  
**ESCALA VETERINÁRIA - MÉDIA**  
**VETERINÄRSKALA – DURCHSCHNITT**  
**ΒΕΤΕΡΙΝΑΡΕΗ ΚΑΙΤΑΡ - СРЕДЕН**  
**VETERINÁRNÍ MĚŘÍTKO - PRŮMĚR**  
**ΚΤΗΝΙΑΤΡΙΚΗ ΚΛΙΜΑΚΑ - ΜΕΣΟΣ ΟΡΟΣ**  
**ELÄINLÄÄKINVÄLINEN ASAKA - KESKIMÄÄRIN**  
**ÁLLATORVOSI MÉRLEG - ÁTLAGOS**  
**SKALA WETERYNARYJNA - ŚREDNIA**  
**SCALA VETERINARĂ – MEDIE**  
**VETERINÁRNA STUPEŇ - PRIEMER**

**REF 6680 (Gima 27258) - 6681 (Gima 27259)**

Manufacturer/Fabbricante/Fabricant/Fabricante/ Hersteller/ производител/  
Výrobce/ Κατασκευαστής/ Valmistaja/ Gyártó/ Producent/ Producător

Výrobca:

**Momert Zrt.**

2400 Dunaújváros - HUNGARY, Papírgyári út 12-14

Made in Hungary


Distribuito da/Distributed by/Distribué par/Distruido por/ Vertrieben von/ Разпространява се от/  
Distribuovaný/Διανέμονται από/ Jakelija/Forgalmazza/ Dystrybuowane przez/ Distribuit de/ Distribuovaný

**Gima S.p.A.**

Via Marconi, 1 - 20060 Gessate (MI) Italy

[gima@gimaitaly.com](mailto:gima@gimaitaly.com) - [export@gimaitaly.com](mailto:export@gimaitaly.com)

[www.gimaitaly.com](http://www.gimaitaly.com)

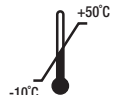

Please study the entire Operating Manual carefully before you start using the product. Keep the manual in a safe place for future reference. Make sure other people using the product are familiar with these instructions.

#### **IMPORTANT SAFETY INSTRUCTIONS:**

- Make sure you follow the marked polarity of battery when inserting and replacing it.
- If the appliance will be out of use for a longer time, remove the battery.
- Do not leave the unit unattended when in use.
- This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capabilities, or lack of experience and knowledge, unless they have been given supervision or instruction concerning use of the appliance by a person responsible for their safety. Children should be supervised to ensure that they do not play with the appliance.
- Use the product only for the intended purpose specified in the Operating Manual.
- Use the unit only on a stable horizontal surface.
- Before cleaning, assembly, or disassembly of the cover, switch the unit off and remove the battery!
- Do not use abrasive or chemically aggressive substances for cleaning the unit.
- Do not use the unit if it does not operate properly, or if it has been dropped, damaged or exposed to any liquids. Take the unit to an authorized service center for testing and repair.
- The unit is suitable for domestic use only. It is not designed for commercial operation.
- Keep the unit away from heat sources, such as radiators, ovens etc. Do not expose the unit to direct sunlight or moisture.
- Do not touch the unit with wet or damp hands.
- The product is a fragile measuring device. Prevent it from crashing or falling.
- Do not repair the appliance yourself. Contact an authorized service center.

#### **FAILURE TO FOLLOW THE MANUFACTURER'S INSTRUCTIONS MAY LEAD TO REFUSAL OF WARRANTY REPAIR!**

#### **SPECIFICATION:**

- Precision measuring system
- Automatic zero resetting
- Automatic switch-off
- Low power indicator
- Overload indicator

#### **TECHNICAL PARAMETERS**

Voltage: 1pc 3V, CR2032 lithium battery (included)

Max. measurable weight: 3-100 kg

Weighing increments: 100 g

Display size: LCD diameter= 8,0 cm, 3,15 inch

Platform: 6680 (GIMA 27258): 635x355 mm; 6681 (GIMA 27259): 830x620 mm

Weighing units: kg, lb, st

Working/Storage temperature: 10-40°C/-10-50°C

#### **THE OPERATION INSTRUCTION**

Before using the scale for the first time, please remove isolating tab from the battery at the backside of the scale.

Place the scale on a flat, hard level surface. On a soft surface ( for example: on rug ) the scale may show inaccurate value.

To start the scale, tap the centre of the platform with your foot.

For the first time the display shows a running "8" and after "0,0" appears on the display.

Wait until "0,0" is displayed!

Place pet on the scale.

Then your pet's weight will appear on the display flashing two times. The measured weight can be seen for about 10 seconds.

If no weighing happens after switching on the scale, it will automatically turn off after 60 seconds.

#### **CHANGE BATTERY**

The battery needs to be replaced, when the display screen shows:



Please open the battery holder at the backside of the scale and carefully take out the battery retracting the black plastic retaining tab. The fastening tab is located at the top of the battery. Do not snap the metal tab. Replace the battery to a new battery. The low battery should be thrown to the appropriate place.

BATTERIES ARE NOT INCLUDED IN WARRANTY!

#### **OVERLOAD WARNING**

When the scale is overloaded, the following is displayed:



#### **CLEANING AND MAINTENANCE**

The surface of the unit may only be cleaned using a wet cloth. Do not use any detergents or hard objects as they may damage the unit's surface!

Never clean the unit under running water, do not rinse it, and do not submerge it in water!

#### **REPAIR AND MAINTENANCE**

Any extensive maintenance or repair that requires access to the inner parts of the product must be performed by an expert service centre.

#### **ENVIRONMENTAL PROTECTION**

- Packaging materials and obsolete appliances should be recycled.
- The transport box may be disposed of as sorted waste.
- Polyethylene bags shall be handed over for recycling.



**Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

#### **GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.