



PROFESSIONAL MEDICAL PRODUCTS

REF 27259/6681

Electronic large pet scale

Technical specification:

1" LCD display
Capacity: 100kg
Graduation: 100g
Auto off
Auto overload indicator ("ERR")
Low battery indicator ("LO")
kg, lb, St unit of measurement can be switched

User Guide

1. Place the scale on a flat, hard level surface. On a soft surface (for example: on rug) the scale may show inaccurate value.
2. To start the scale, tap the scale platform with your foot.
3. For the first time the display shows a running "8" and after "0,0" appears on the display.
4. Wait until "0,0" is displayed!
5. Place pet on the scale.
6. Then your pet's weight will appear on the display flashing two times. The measured weight can be seen for about 10 seconds.
7. After weighing, it will automatically turn off approximately in 60 seconds.

Power:

The scale run on replaceable Lithium battery. (CR 2032) included

Replacing batteries:

When "LO" appears on the display, replace the batteries.

Error display:

Weighing above 100 kg, "ERR" is displayed.
Please do not overload the scale! In case of overloading the scale get damaged. In this case we have no liability for the product.

Environment:

The environmental temperature may influence the weighing accuracy. You can get correct weighing accuracy at working temperature (10-40°C)

Remarks:

Please do not use the scale in the immediate vicinity of strong emitters such as mobile phones or microwave ovens. To clean the scale use damp clothes without any solvent.



Disposal: The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.

Manufacturer/Produttore/Fabricant/Fabricante:

Momert Co.Ltd.

2400 Dunaújváros - HUNGARY, Papírgyári út 12-14

Made in Hungary



Distribuito da/Distributed by/Distribué par/Distribuido por

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

M 27259 - EN - Rev. 2.02.2020