

Thank you for choosing this Soehnle Professional product. This product has all the features you'd expect from the latest technology and is designed for ease of operation.

If you have any questions or if you experience problems with your baby scale that are not dealt with in the operating instructions, please contact your Soehnle Professional service partner or visit our website www.soehnle-professional.com.

Purpose

The Soehnle Professional Baby Scale is designed for weighing infants. This scale intended to be used without requiring approval. It complies with the relevant requirements of EU directives 2004/108/EC and 2006/95/EC.

Safety instructions

Please read the information contained in the operating instructions before starting the device. This provides important advice on the installation, correct usage and maintenance of the device.

The manufacturer accepts no liability if the following instructions are not complied with. The devices are designed to be used indoors. The scale meets the requirements for electromagnetic compatibility. Avoid exceeding the maximum values specified in the standards. If problems arise, please contact your Soehnle Professional service partner.

Cleaning

A damp cloth and commercially available cleaning agent are all that is required. Do not use abrasive cleaners.

Maintenance

The technical control of measuring accuracy was carried out by the manufacturer. The scale does not require routine maintenance. However, it is advisable to check its accuracy at periodic intervals. The frequency of such checks depends on the frequency of use and the state of the scale. If you find any discrepancies (at 15 kg \pm 50 g) you should contact your dealer or the Soehnle Professional service partner. Repairs may only be carried out by workshops authorized by Soehnle Professional using original parts.

Note

This device is interference-free in accordance with the relevant EU Directive 2004/108/EC. However, under extreme electrostatic an electromagnetic interference, e.g. when operating a radio or mobile phone in the immediate vicinity of the device, the displayed value can be falsified. After the interference has ended, the product can be used again as intended, though it may be necessary to switch it on again.

The device is a measuring instrument. Drafts, vibrations, rapid changes in temperature and direct sunlight can effect the weighing results.

The scale complies with protection class IP 54. High levels of humidity, vapor, aggressive liquids and heavy dirt should be avoided.

Warranty/liability

If there is a defect in the material delivered for which Soehnle Professional is responsible, Soehnle Professional is entitled either to eliminate the defect or to provide a replacement. Replaced parts become the property of Soehnle Professional. If the repair or replacement fail to achieve satisfactory results, the relevant legal provisions shall apply. The warranty period is 2 years and starts on the day of purchase. Please keep the invoice as proof. If you require service, please contact the dealer or Soehnle Professional Customer Service.

In particular, no warranty is offered on damage arising for the following reasons: unsuitable, incorrect storage or use, faulty installation or commissioning by the client or by a third party, natural wear, modifications or interventions, incorrect or negligent handling, particularly excessive stress, chemical, electro-chemical or electrical influences or humidity, insofar as these cannot be traced back to fault on the part of Soehnle Professional. If operational, climatic or other influences lead to major changes in conditions or in the state of the material, the warranty for the correct overall function of the devices shall no longer apply. If Soehnle Professional offers a guarantee on an individual basis, this means freedom from defects on the object delivered in the guarantee period. Keep the original packaging in case it is necessary to return the scale.

Disposing of the scale

According to our current level of knowledge, the Baby Scale does not contain any special materials that are hazardous to the environment.

This product should not be treated as normal refuse, but should be returned to a collection point for recycling electrical and electronic devices. For further information, contact your local authority, the local waste disposal businesses or the company from whom you purchased the product.

Disposing of batteries: Batteries do not belong with domestic waste. As a consumer you are obliged by law to return used batteries. You can return your used batteries to the public collection points in your locality or wherever batteries of the relevant type are sold. You will find the following codes on batteries that contain hazardous materials: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

Soehnle Professional GmbH & Co. KG
Manfred-von-Ardenne-Allee 12 · DE-71522 Backnang
Phone +49 (0) 71 91 / 34 53-0 · Fax - 211
www.soehnle-professional.com

