
seca viva 750/754

seca colorata 760

seca medica 761/762

seca selecta 791

seca artista 830

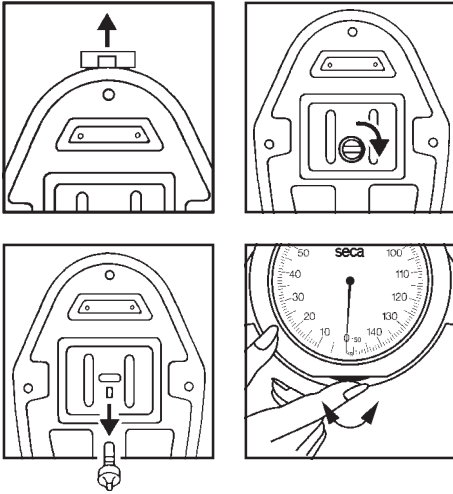Precision for Health

With this purchase, you have decided on precision, reliability and a good design. From the leading manufacturer of medical scales with over 150 years experience. Today, however, our slogan *seca* Precision for Health includes much more: Medical scales and scales for the private use.

And this is how it functions

- Remove the transportation protection. Twist the red protecting pin which can be found underneath the scale, and then pull out. Following this, simply remove the red square protecting piece.
- Set the display to zero. The small wheel can be found directly below the zero.

Care:

Use normal cleaning agents. Do not use scouring agents.

The scale ring glass can be cleaned from the inside when removed (*seca* 760, 761/762, 791).

Just turn the ring to the left and then remove. Place back in position and turn to the right.

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Warranty

A three-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was

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when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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