



GIMA

PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A.
Via Marconi, 1 - 20060 Gessate (MI) Italy
gima@gimaitaly.com - export@gimaitaly.com
www.gimaitaly.com

PESANEONATI FAMILY FAMILY SCALE BALANCE FAMILY FAMILIENWAAGE BÁSCULA FAMILY BALANHA FAMILY ΖΥΓΑΡΙΑ FAMILY FAMILY الميزان

Manuale d'uso - User manual - Manuel de l'utilisateur
Gebrauchs- und instandhaltungsanleitung
Guía de uso - Guia para utilização
Οδηγίες χρήσης - دليل الإستعمال والرعاية



ATTENZIONE: Gli operatori devono leggere e capire completamente questo manuale prima di utilizzare il prodotto.

ATTENTION: The operators must carefully read and completely understand the present manual before using the product.

AVIS: Les opérateurs doivent lire et bien comprendre ce manuel avant d'utiliser le produit.

ACHTUNG: Diese Anleitung muss vor dem Einsatz des Produkts aufmerksam gelesen und vollständig verstanden werden.

ATENCIÓN: Los operadores tienen que leer y entender completamente este manual antes de utilizar el producto.

ATENÇÃO: Os operadores devem ler e entender completamente este manual antes de usar o produto.

ΠΡΟΣΟΧΗ: Οι χειριστές αυτού του προϊόντος πρέπει να διαβάσουν και να καταλάβουν πλήρως τις οδηγίες του εγχειριδίου πριν από την χρήση του.

الحذر: على العمال قراءة وفهم هذا الدليل بكامله قبل البدء باستعمال المنتج.




REF 27301



Gima S.p.A.
Via Marconi, 1
20060 Gessate (MI) Italy
Made in China

CE 0476



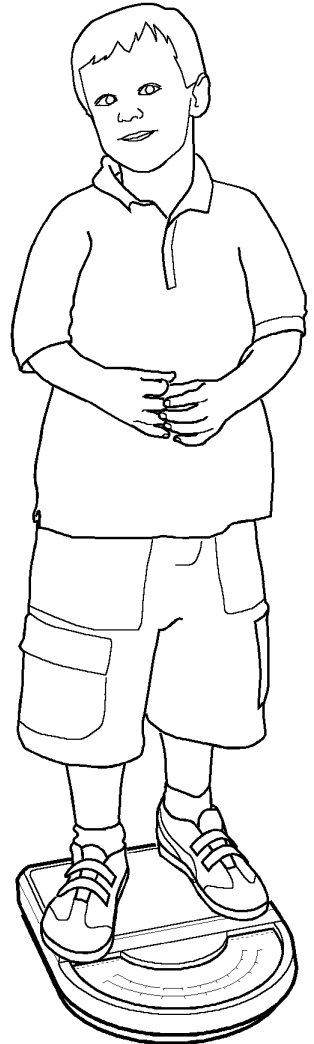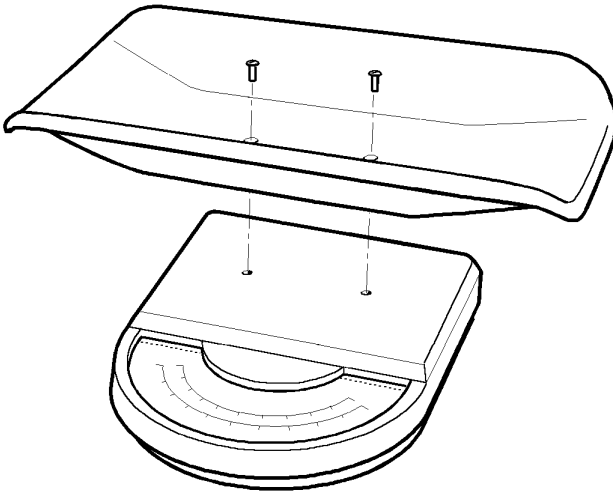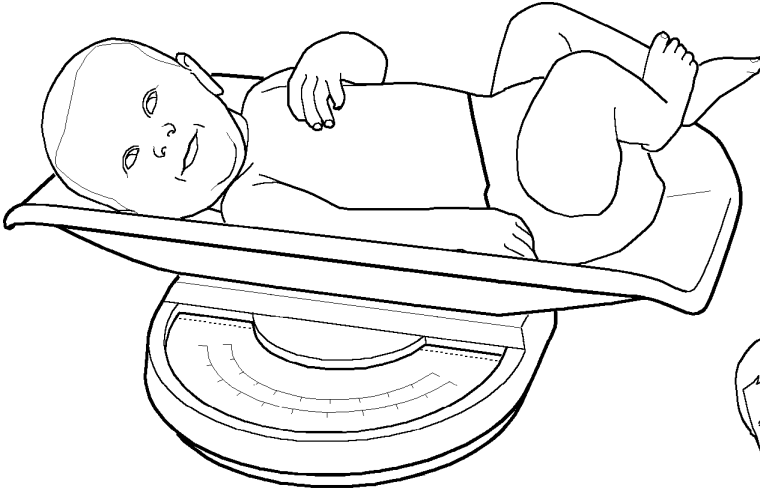



FEATURES

The GIMA FAMILY scale is a device used to weight babies. The scale is designed for home use and includes a removable tray. Manual zero setting.

Technical specifications

Maximum load: 20 Kg / 44lb

Accuracy: 50 g (0-10 Kg), 100g (10-20 Kg)

Weight of machine: 3.0 Kg

PRESCRIPTIONS



Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING



Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled.

They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

After unpacking the scale, assemble the tray by fixing the two fastening screws with a screwdriver. There is no need to tare the scale. It has to be set on a flat surface (table or floor) for correct operation.

OPERATION

Before weighing the baby check the position of the needle, if the needle is not on the zero bring it to zero by rotating the adjusting knob. After adjusting the scale, slightly press down on the tray several times (1-2 Kg pressure). If after pressing the needle goes back to zero, the scale is ready to be used.

If the child can stand, you can use the scale without the tray. In this case, let the child stand directly on the base of the scale.

When the tray is removed, adjust the scale again. The needle has to be set to zero!

MAINTENANCE

The product does not require any special maintenance. The technical life of the scale depends on its use. It is resistant against most chemical products except for M.E.K. ketons, diluting substances and polish removers.









GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use.

GIMA cannot be held responsible for malfunctioning on electronic devices or software due to outside agents such as: voltage changes, electro-magnetic fields, radio interferences, etc.

The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.

**Indice dei Simboli / Index of symbols / Liste des symboles/ Symbole / Índice de simbolos /
 Índice de Símbolos / Πίνακας συμβόλων / حرف**

	<p>IT - Leggere e seguire attentamente le istruzioni per l'uso GB - Read instructions carefully FR - Lire et suivre attentivement la notice DE - Gebrauchsanweisung sorgfältig lesen und befolgen ES - Leer y seguir atentamente las instrucciones de uso</p>	<p>PT - Ler e seguir atentamente as instruções de uso GR - Διαβάστε και ακολουθήστε προσεκτικά τις οδηγίες χρήσης SA - اقرأ بدقة وحرص تعليمات الاستخدام</p>
	<p>IT - Conservare in luogo fresco ed asciutto GB - Keep in a cool, dry place FR - À conserver dans un endroit frais et sec. DE - An einem kühlen und trockenen Ort lagern ES - Conservar en un lugar fresco y seco</p>	<p>PT - Armazenar em local fresco e seco GR - Διατηρείται σε δροσερό και στεγνό περιβάλλον SA - يحفظ في مكان بارد وجاف</p>
	<p>IT - Fabbricante GB - Manufacturer FR - Fabricant DE - Hersteller ES - Fabricante</p>	<p>PT - Fabricante GR - Παραγωγός SA - الشركة المصنعة</p>
	<p>IT - Prodotto conforme alla Direttiva Europea GB - Product complies with European Directive FR - Ce produit est conforme à la directive européenne DE - Produkt gemäß EU-Richtlinie ES - Producto conforme a la Directiva Europea</p>	<p>PT - Produto em conformidade com a Diretiva Europeia GR - Προϊόν σύμφωνο με την Ευρωπαϊκή Οδηγία SA - منتج يتوافق مع توجيهات الاتحاد الأوروبي رقم</p>
	<p>IT - Codice prodotto GB - Product code FR - Code produit DE - Erzeugniscode ES - Código producto</p>	<p>PT - Código produto GR - Κωδικός προϊόντος SA - كود المنتج</p>
	<p>IT - Conservare al riparo dalla luce solare GB - Keep away from sunlight FR - À conserver à l'abri de la lumière du soleil DE - Vor Sonneneinstrahlung geschützt lagern ES - Conservar al amparo de la luz solar</p>	<p>PT - Guardar ao abrigo da luz solar GR - Κρατήστε το μακριά από ηλιακή ακτινοβολία SA - يحفظ بعيداً عن ضوء الشمس</p>
	<p>IT - Leggere attentamente le istruzioni per l'uso GB - Please read instructions carefully FR - Lire et suivre attentivement la notice DE - Gebrauchsanweisung sorgfältig lesen ES - Leer atentamente las instrucciones de uso</p>	<p>PT - Ler atentamente as instruções de uso GR - Διαβάστε προσεκτικά τις οδηγίες χρήσης SA - اقرأ بدقة وحرص تعليمات الاستخدام</p>
	<p>IT - Numero di lotto (vedi scatola / bustina) GB - Lot number (see box / package) FR - Numéro de lot (voir boîte/sachet) DE - Chargennummer (siehe Schachtel / Beutel) ES - Número de lote (ver caja / sobre)</p>	<p>PT - Número de lote (ver caixa/saqueta) GR - Αριθμός παρτίδας SA - رقم اللوتل/رقم التشغيلية</p>