



GIMA

PROFESSIONAL MEDICAL PRODUCTS

BILANCIA PER NEONATI

BABY SCALE

PÈSE-BÉBÉ

BÁSCULA PARA BEBÉS

BALANÇA DE BEBÉ

BABYWAAGE

لفظلا نازيم

Manuale d'uso - User manual - Mode d'emploi - Manual de instrucciones - Manual de Operação - Bedienungsanleitung -
مادختسالا لي ليد

Modello/Model: RGZ-20A (GIMA 27302)

Fabbricante / Manufacturer / Fabricant / Fabricante / Fabricante / Hersteller / غناصلا

Wuxi Weigher Factory (Limited Company)

Majiaqiao, Guangfeng, Guangyi – 214011 Wuxi, Jiangsu

POPLE'S REPUBLIC OF CHINA - Made in China

Importato e distribuito da / Imported and distributed by / Importé et distribué par / Importado y distribuido por
Importado e distribuido por / Importiert und vertrieben von / مت ليق نم اهعيزوتو اهداريتسا /

Gima S.p.A.

Via Marconi, 1 - 20060 Gessate (MI) Italy

gima@gimaitaly.com - export@gimaitaly.com

www.gimaitaly.com

Please read this manual carefully before using scale

Specifications

Type	RGZ-20A-YE
Max.	20kg/44lb
Division	50g (0~10kg) /2oz(0~22lb) 100g (10~20kg) /4oz(22~44lb)
Plate Dimensions	545X270 (LXW)
Overall Dimensions	545X370X150 (LXW XH)
Net Weight	3.1kg

PRESCRIPTIONS

Do not use the equipment in case it is damaged. Apply to your retailer.

Avoid precarious repairs. Repairs shall be carried out with original spare parts only, which shall be installed according to the intended use.

Since the product is made of corrosion-proof materials suitable for the environmental conditions foreseen for its normal use, does not require special care, however it is necessary to store it in a closed place making sure that is protected from dust and dirt to assure its hygienic conditions. Moreover, it is recommended to store the product in a place which can be reached easily by the personnel in case of necessity.

UNPACKING

Always remember that packing elements (paper, cellophane, stitches, adhesive tape, etc.) can cut and/or hurt if they are not carefully handled. They shall be removed with adequate means and shall not be left at the mercy of irresponsible persons; the same is valid for tools used to remove packages (scissors, knives, etc.).

After opening the packages, first of all it is necessary to check all pieces and parts composing the product. Check that they are all present and in perfect conditions.

INSTALLATION

After unpacking the scale, assemble the tray by fixing the two fastening screws with a screwdriver. There is no need to tare the scale. It has to be set on a flat surface (table or floor) for correct operation.

OPERATION

Before weighing the baby check the position of the needle, if the needle is not on the zero bring it to zero by rotating the adjusting knob. After adjusting the scale, slightly press down on the tray several times (1-2 Kg pressure). If after pressing the needle goes back to zero, the scale is ready to be used.

If the child can stand, you can use the scale without the tray. In this case, let the child stand directly on the base of the scale. When the tray is removed, adjust the scale again. The needle has to be set to zero!

MAINTENANCE

The product does not require any special maintenance. The technical life of the scale depends on its use. It is resistant against most chemical products except for M.E.K. ketenes, diluting substances and polish removers

 **Disposal:** The product must not be disposed of along with other domestic waste. The users must dispose of this equipment by bringing it to a specific recycling point for electric and electronic equipment.

GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies.