# GIMA

PROFESSIONAL MEDICAL PRODUCTS

**MISURATORE CORPOREO**

**BODY METER**

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**MEDIDOR CORPORAL**

**KÖRPERFETTMESSGERÄT**

**MEDIDOR CORPORAL**

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Fabbricante/Manufacturer/Fabricant/Fabricante/Fabricante/Hersteller/Κατασκευαστής/عناصلا:  
Samson Innovation Corporation Ltd. - Room F, 8th Floor, Hop Hing Industrial Building, 704 Castle Peak Road,  
Lai Chi Kok, Kowloon, Hong Kong - Made in China

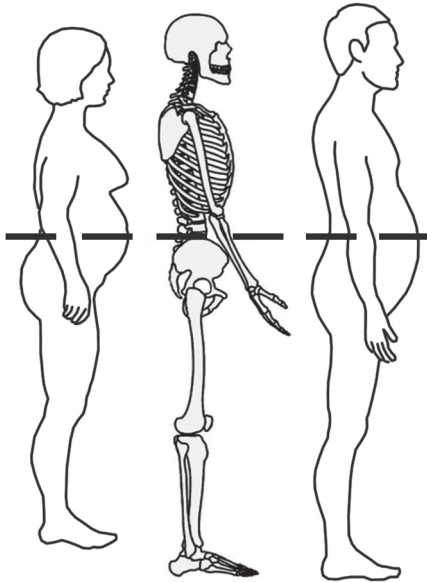
Tape to measure the distribution of adipose tissue. ABS body with ergonomic grip. The tape is made of a special non-deformable plastic material, with double coloured measurement scale for separate risk indications, male and female.

It can be used on all body circumferences, thanks to its retractable button and flexible stop which allows for the data to be read easily.

The particular flexibility of its non-toxic thermoplastic arm allows for specific use in paediatric settings

Tape position for the correct measurement of the abdominal circumference

*Individual in standing position.*

*Locate the upper edge of the hip and the top of the right iliac crest.*

*Place the horizontal tape around the abdomen at the level of the edge of the iliac crest.*

*Before taking the measurement, make sure that the tap is tight but not compressing the skin and that it is parallel to the floor.*

*Take the measurement at the end of a normal exhalation.*

### **GIMA WARRANTY TERMS**

The Gima 12-month standard B2B warranty applies.

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**Gima S.p.A.** - Via Marconi, 1 - 20060 Gessate (MI) Italy
**Solution Group S.r.l.** - Via Carlo Sormani, 1 - 20095 Cusano Milanino (MI) - Italy