



# GIMA

PROFESSIONAL MEDICAL PRODUCTS

## **LETTINO VISITA MEDICA TRADIZIONALE EXAMINATION COUCH - ALUMINIUM**

**MANUALE D'USO – ISTRUZIONI E MONTAGGIO  
USER MANUAL – ASSEMBLY INSTRUCTIONS**

È necessario segnalare qualsiasi incidente grave verificatosi in relazione al dispositivo medico da noi fornito al fabbricante e all'autorità competente dello Stato membro in cui si ha sede

All serious accidents concerning the medical device supplied by us must be reported to the manufacturer and competent authority of the member state where your registered office is located

**REF** 27403



Gima S.p.A.  
Via Marconi, 1 - 20060 Gessate (MI) Italy  
gima@gimaitaly.com - export@gimaitaly.com  
[www.gimaitaly.com](http://www.gimaitaly.com)  
Made in Italy



## INTRODUCTION

For the purposes of Regulation (EU) 2017/745, the product falls under the CLASS I MEDICAL DEVICES category. We advise you to read the following manual carefully before using the device.

This model has been designed with the aim of limiting any risks to the user in the event of unintentional impact, by eliminating sharp edges, sharp corners, etc.

## SAFETY

Use for purposes other than what it is intended for is not allowed.

The manufacturer declines all responsibility for damage caused by using the product the wrong or improper way, or in a way unforeseen in the current manual.

For safety reasons, replace any worn or damaged parts immediately.

## MAINTENANCE / CLEANING

The 1-joint examination bed does not require any special maintenance; however, before use, it is necessary to check that all the moving parts are tightened correctly.

To clean the bed, a slightly damp cloth can be used; normal detergents can also be used, provided that they do not contain chlorine and its compounds.

Attention:

- Avoid using hydrogen chloride or bleach.

## ASSISTENCE

Should any of the component parts deteriorate beyond general maintenance, contact your distributor, they will be available to advise you on the most appropriate solution to every kind of problem.

## WARNINGS

The information contained in this document are subject to modifications without notice having to be given.

Disposal: the product described in this manual must be sent to specific companies for scrapping.

## GENERAL INFORMATION


Examination bed with a frame made of round polished aluminium tubes Ø 30 x 2 mm thick; fitted with an elevating, manually adjustable headrest with 1 joint and 2 sections, and upholstered in soft, easily washable faux leather. The upholstery consists of: chipboard base with low formaldehyde content, helio-cell sponge with a density of 25 kg/sq m, 4 cm thick, convenient and comfortable.

Unassembled model supplied as a kit







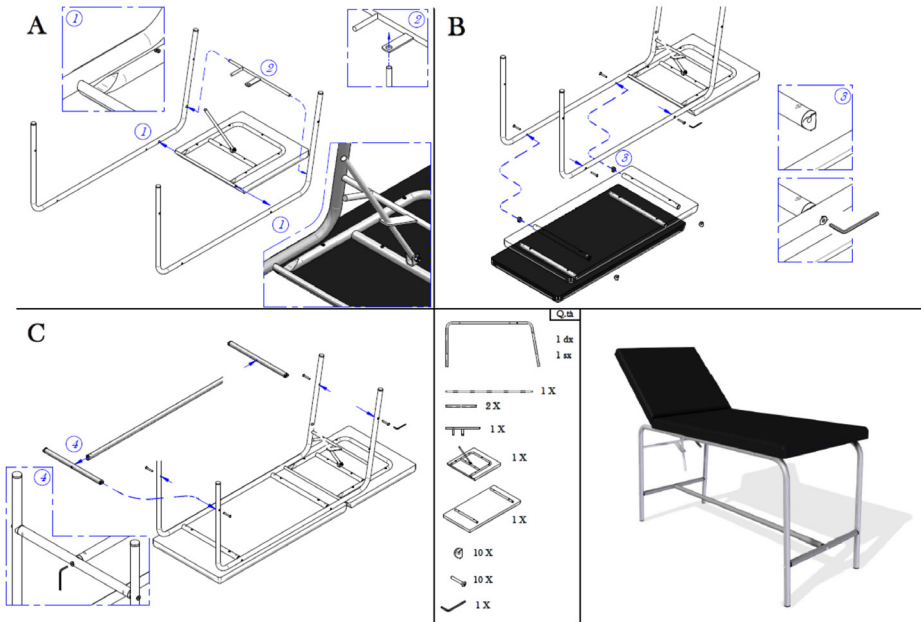



## TECHNICAL DATA SHEET

Dimension (L x P x H) cm	176 x 60 x 78
Capacity Kg	90
Net weight kg	22.0
Gross weight kg	27.0
Volume M3	0.21
Number of necks	1
Packaging (LxDxH) Cm	151 x 19 x 74

## ASSEMBLING INSTRUCTIONS



### Symbols:

	Caution: read instructions (warnings) carefully		Keep away from sunlight
	Keep in a cool, dry place		Date of manufacture
	Manufacturer		Product code
	Lot number		Consult instructions for use
	Medical Device compliant with Regulation (EU) 2017/745		Medical Device

### GIMA WARRANTY TERMS

The Gima 12-month standard B2B warranty applies